

DIETZ has developed a dedicated tool to help you adjust the bag suspension. This tool is shaped like the DIETZ logo and is supplied with the rollator. The following chapter describes how to use the tool for adjusting the bag suspension.

ADJUSTING THE BAG SUSPENSION



WARNING

Magnetic bag suspension: Follow the user manual and safety information for your cardiac pacemaker.



NOTICE

Do not fully unscrew the bag suspension.

The supplied tool lets you adjust the bag suspension in case it is not adjusted to accept the bag perpendicularly from above.

1. To engage the parking brakes, push the brake levers down.
2. Unfold the rollator.
3. In case the bag is attached, remove it.
4. Hold the DIETZ logo side of the tool in your hand.
5. Place the tool on the bag suspension in such a way that it engages **(1)**.
6. Rotate the bag suspension using the tool **(2)** so that its opening faces upwards **(3)**.
7. Repeat steps 4 – 6 on the other side of the bag suspension if required.

