
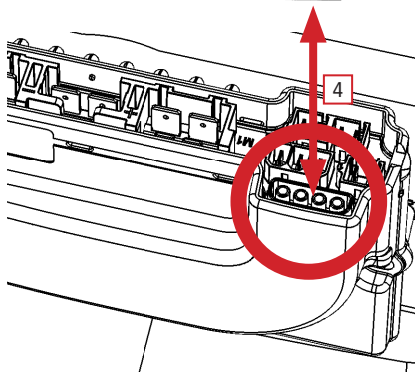


Versand/Transport Joystick trennen

Drücken Sie mittig auf die Oberkante [1] der Abdeckung um sie zu entriegeln, dann nach vorne herunterklappen [2] und die Abdeckung entnehmen.

Der Stecker hat einen Sicherungshebel [3] welcher beim Herausziehen gedrückt sein muss. Die Steckerverbindung des Joysticks am Controller trennen [4].

Die Abdeckung an der Unterseite in die Aufnahmen einhängen, hochklappen und durch mittigen Druck auf der Abdeckungsobenseite den Klemmriegel der Abdeckung in den Rollstuhlrahmen einrasten lassen.

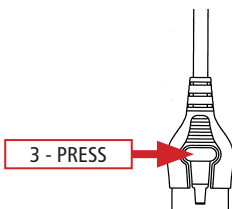

Shipping/Transport Joystick disconnect

The cover must be removed by depressing on the cover's middle top section [1], rotate down [2] and release.

The joystick's connector has a safety [3] which must be pressed down to release the plug from the controller.

Disconnect the joystick's plug from the controller [4].

Assemble the cover after connecting the joystick's plug back into the controller. Place the cover on the chassis hinges and rotate it upwards, depress on the middle section, push in until the cover is locked into the chassis.