
DEU

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ENG

LIAM SEATING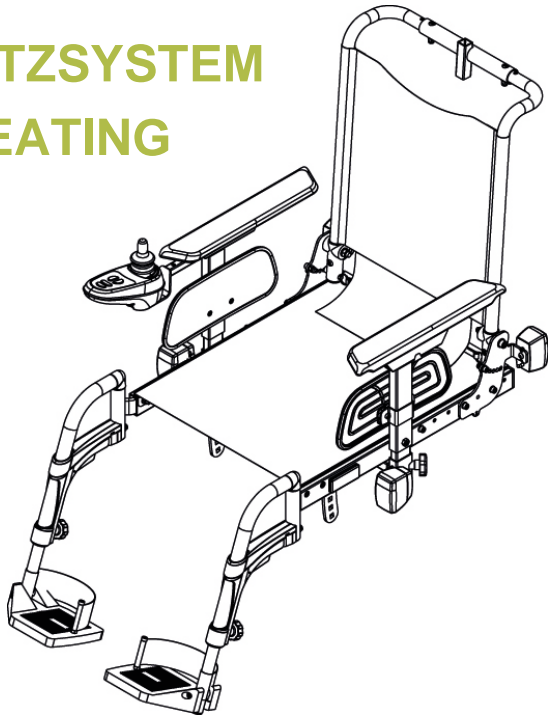

BEDIENUNGSANLEITUNG
USER MANUAL

6003078A [8K10]

Deutsch

© 2014 YOU-Q

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© 2014 YOU-Q

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of YOU-Q.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. YOU-Q executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. YOU-Q cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but YOU-Q cannot be held liable for possible errors in the information or the consequences thereof. YOU-Q accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by YOU-Q may not, as per the legislation concerning the protection of trade names, be considered as being available.

2014-07

Table of content

1	Foreword	18
2	General description	18
2.1	Product description Liam Seating	18
3	General operations	19
3.1	Legrest: Removing / replacing	19
3.2	Armrest: Removing / replacing.....	19
3.3	Headrest	19
4	Setting the seating system	20
4.1	Seat depth setting	20
4.2	Seat width setting.....	20
4.3	Backrest angle setting.....	21
4.4	Legrest settings.....	21
4.5	Armrest settings	22
4.6	Headrest	23
4.7	Hip belt.....	24
5	Adjustments	Fehler! Textmarke nicht definiert.
5.1	Electrical adjustments	Fehler! Textmarke nicht definiert.
6	Getting in and out of the wheelchair (transfer)	25
7	Reducing the wheelchair size	26
8	Maintenance	26
8.1	Maintenance table.....	26
8.2	Cleaning the upholstery	26

1 Foreword

This user manual

The user manual for this electric wheelchair consists of three booklets.

Read the entire user manual carefully before taking the product into use. The information provided by this manual is essential for the safe use and proper care (cleaning) of the wheelchair.

The general user manual (this booklet)
The user manual for the seating system (Liam)

The user manual for the controller

If one of these user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

This general user manual will refer, where necessary, to one of the other user manuals. This is indicated as follows:

 **GENERAL**

 **CONTROLLER:** Refers to the user manual for the controllers.

 **Strictly follow the procedures with a caution symbol! These procedures can if they are not executed properly - result in human injury or damage to the product or the environment.**

CE declaration of conformity

The product is in conformity with the provisions of the Medical Devices Directive and thus has CE marking.

Warranty

The warranty stipulations applicable to the seating system are identical to and incorporated in the wheelchair manual.

2 General description

2.1 Product description Liam Seating

The wheelchair is equipped with the Liam seating system. The Liam seating system provides excellent support for the head, torso, arms and legs. Optimal body support ensures maximum independence for the wheelchair user. The wheelchair must be adjusted by a qualified specialist. Incorrect settings may have negative consequences for the sitting posture.

Please study this manual carefully before you use the wheelchair.

3 General operations

3.1 Legrest: Removing / replacing

Removing (fig 3.1).

1. Push the catch(1) upwards in order to unlock the leg rest.
2. Swing the leg rest outwards.
3. Remove the leg rest from its bracket by lifting.

 **You should hear a click when replacing the leg rest. This indicates that the leg rest has locked into place.**

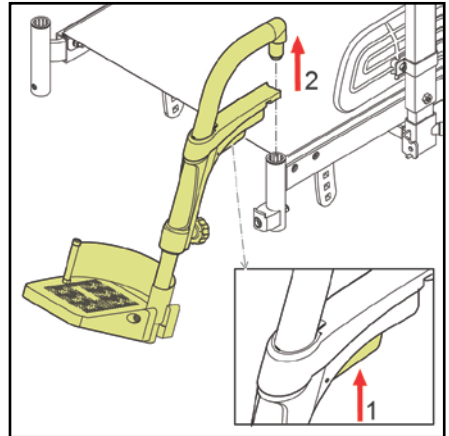
Figure 3.1

3.2 Armrest: Removing / replacing

Removing the armrest (fig 3.2).

1. Loosen the bolts.
2. Take the armrest out.

Replacing the armrest (fig 3.2).

1. Place the armrest into the armrest holder.
2. Tighten the bolts.

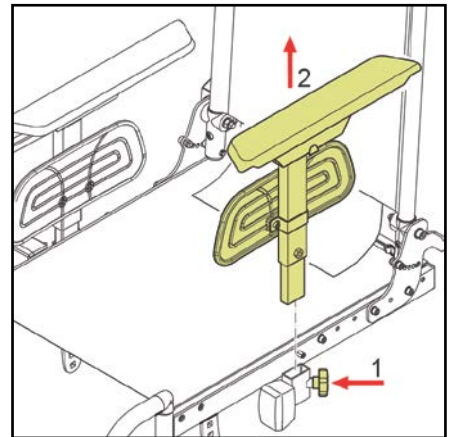
Figure 3.2

3.3 Headrest

Removing the headrest (fig 3.3).

1. Loosen the star knob.
2. Take the headrest out.

Replacing the headrest (fig 3.3).

1. Place the headrest into the head rest holder.
2. Tighten the star knob.

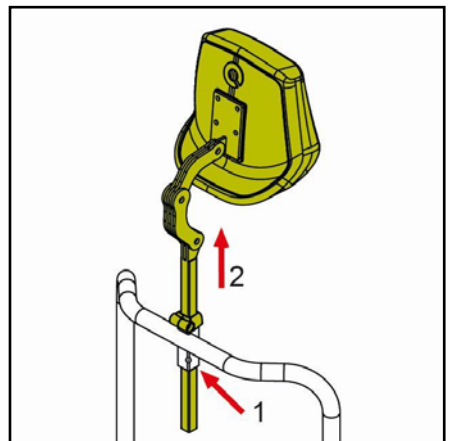
Figure 3.3

4 Setting the seating system

4.1 Seat depth setting

Seat depth by backrest frame (fig 4.1).

1. Loosen the bolts on both sides.
2. Place the backrest frame in the desired position.
3. Tighten the bolts.

 **Make sure that there is at least 2 cm of free space between the back of the knee and the front of the seat to ensure good blood circulation.**

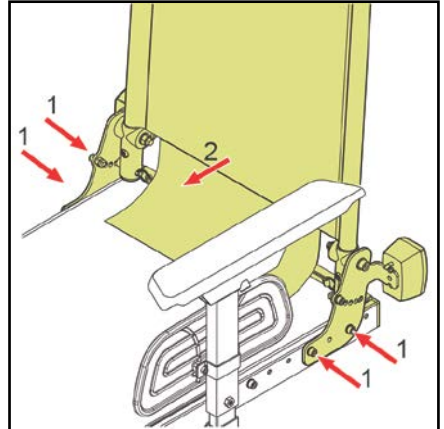
Figure 4.1

4.2 Seat width setting

Seat width setting by armrests (fig 4.2).

1. Loosen the bolts.
2. Slide the armrest in the desired position.
3. Tighten the bolts.

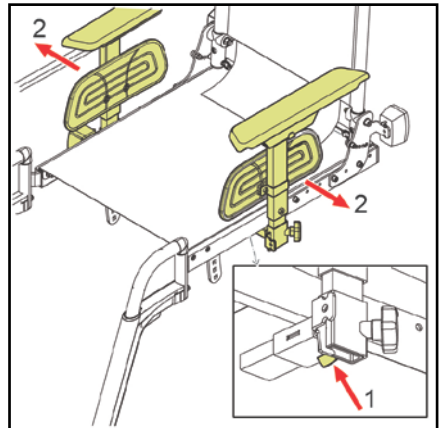
Figure 4.2

4.3 Backrest angle setting

See figure 4.3

1. Loosen the joints (1) by drawing the strap behind the backrest.
2. Set the desired angle of the backrest.
3. Take care that both pins left and right, fit into the locking holes.

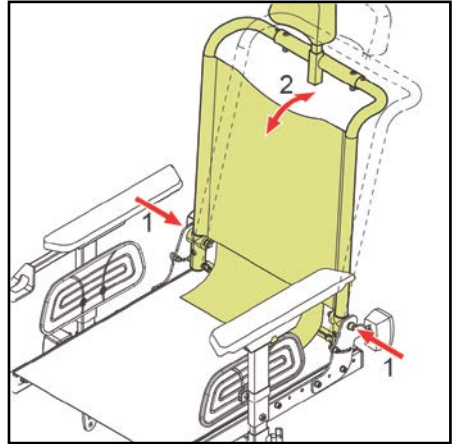
Figure 4.3

4.4 Legrest settings

Lower leg length (fig 4.4).

1. Loosen the star knob.
2. Slide the footplate in the desired position.
3. Tighten the star knob.

 **Make sure that there is sufficient free space under the footplates to operate the wheelchair over obstacles.**

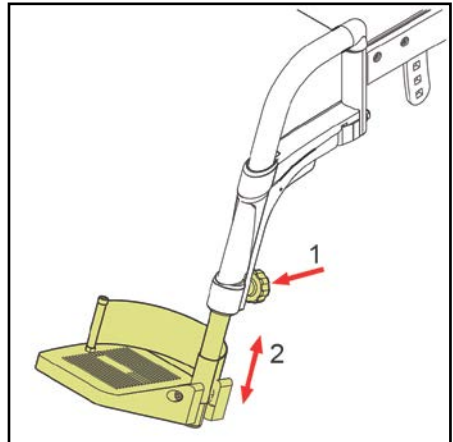
Figure 4.4

4.5 Armrest settings

Armrest height setting (fig 4.5).

1. Loosen the star knob.
2. Adjust to the desired height.
3. Tighten the star knob.

 **Make sure that the arms are supported well, without the shoulders being pushed upward.**

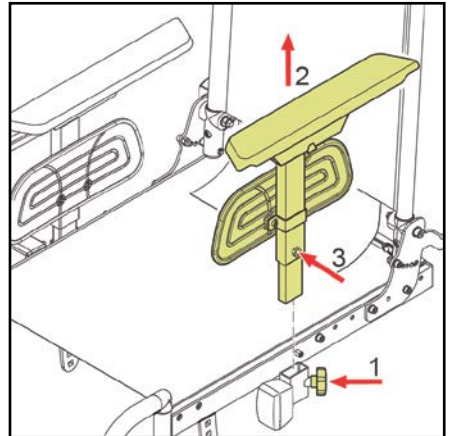
Figure 4.5

Sideplate height setting (fig 4.6).

1. Loosen the bolts.
2. Slide the armrest in the desired position.
3. Tighten the bolts.

Armrest width setting (fig 4.2).

See chapter 4.2

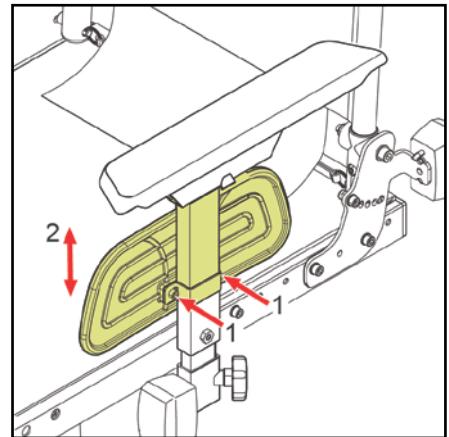
Figure 4.6

Controller depth setting (fig 4.7).

1. Loosen the bolt.
2. Slide the control in the desired position.
3. Tighten the bolt.

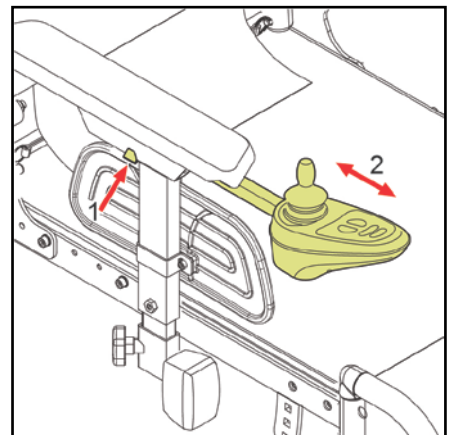
Figure 4.7

4.6 Headrest

Height (fig 4.8).

1. Loosen de starknob.
2. Slide the headrest in the desired position.
3. Tighnten de starknob.

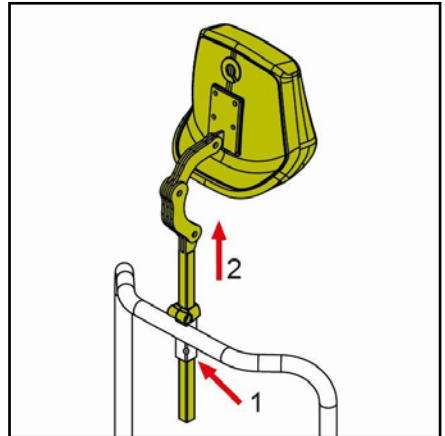
Figure 4.8

Memory block (fig 4.9).

1. Loosen the bolt.
2. Slide the memory block in the desired position.
3. Tighten the bolt.

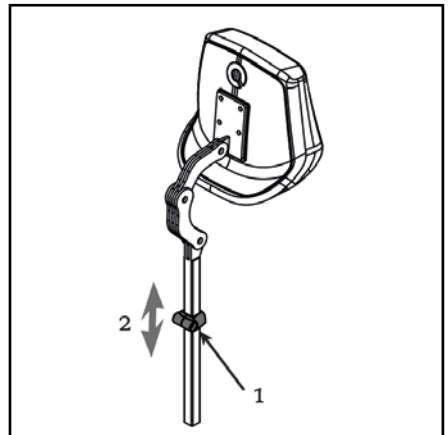
Figure 4.9

Position (fig 4.10).

1. Loosen the bolts (x3).
2. Put the headrest in the desired position.
3. Tighten the bolts (x3).

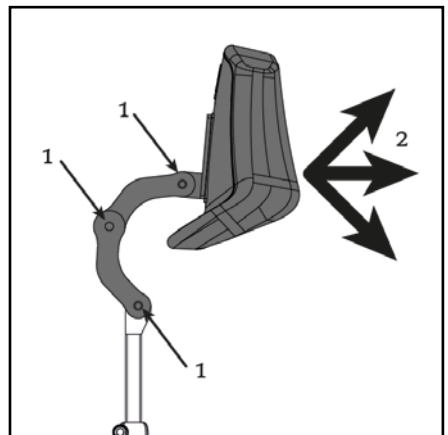
Figure 4.10

4.7 Hip belt

It is possible for the wheelchair to be fitted with a positioning belt. The hip belt offers extra seating stability under normal use.

The hip belt is attached as follows (figure 4.11):

1. Attach the belt buckle using a nut and bolt.
2. Pass the belt through the buckle.

- **Check that the hip belt is correctly adapted to the wheelchair user. The belt should lie firmly over the hip at an angle of approx. 45° (fig 4.12).**
- **Placing the belt over the hip, against the soft skin of the stomach, can result in the wrong sitting position and the user sliding forward in the wheelchair (figure 4.13).**
- **Make sure the belt is clean and that the locking mechanism works at all times. The belt and locking mechanism can be cleaned using a damp cloth.**
- **Never use the hip belt as a safety belt when transporting the user in the wheelchair.**

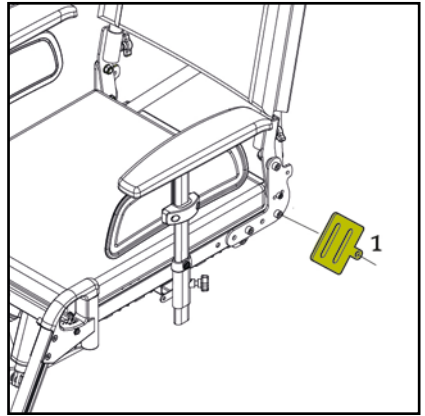
Figure 4.11

Figure 4.12: Right position hip belt

Figure 4.13: Wrong position hip belt

5 Getting in and out of the wheelchair (transfer)

 Do not stand on the foot-plates. These have not been designed to accommodate the full weight of a person. In addition, this could cause the wheelchair to topple over.

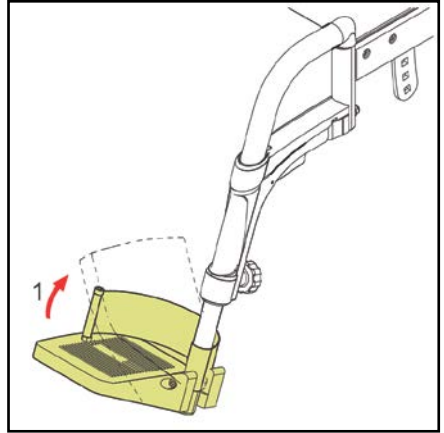
Figure 6.1

Forward transfer

1. Flip the footplates up (fig 6.1)..
2. If possible, swing the legrests to the side (fig 6.2).
3. Seat yourself in the chair.

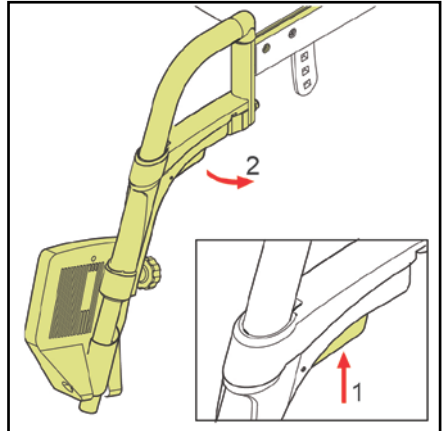
Figure 6.2

Sideways transfer (fig 6.3).

1. Loosen the bolt.
2. Take the armrest out.
3. Seat yourself in the chair.

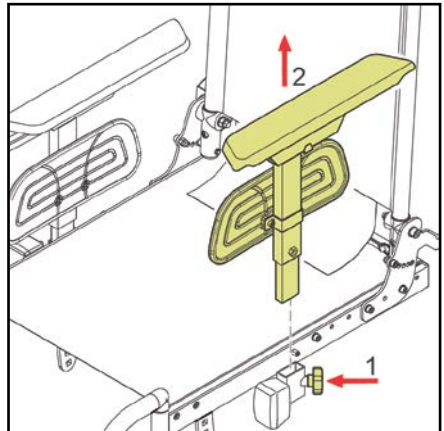
Figure 6.3

6 Reducing the wheelchair size

The wheelchair can be reduced in size for transportation without a passenger. Any components of the wheelchair that can easily be detached should be removed. Store such components in a safe place. The backrest can also be folded down if the car's boot space is small.

1. Remove the seat cushion. See ch. 3.
2. Remove the legrest. See ch. 3.
3. Remove the armrests. See ch. 3.
4. Remove the headrests. See ch. 3.
5. Flip the backrest frame down. See ch. 3.

7 Maintenance

7.1 Maintenance table

It is recommended to have your wheelchair serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Monthly	Cleaning the upholstery, if necessary. See 8.2.

7.2 Cleaning the upholstery

Clean the upholstery with a damp cloth and household soap. Then wipe the upholstery dry with a soft dry cloth.

Do not use organic solvents such as thinner, benzene or white spirit. Do not dry clean, iron or spin-dry the upholstery

YOU-Q B.V.
Lagedijk 31
5705 BX Helmond

T +31(0)492 50 57 00
F +31(0)492 50 57 01
info@you-q.nl
www.you-q.nl