D/GB

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Translation of the original user's manual

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CE-Declaration of conformity

The company AAT Alber Antriebstechnik GmbH herewith declares that the attendant control of the SOLO is conform to the current regulations of the European Union, EEC Guidelines 2006/42/EG, 2007/47/EG and 93/42/EG. This declaration becomes void, if the attendant control has been altered without the consent of AAT Alber Antriebstechnik GmbH.

Signature of the manufacturer:

Martin Kober
(Managing shareholder)

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The attendant control for the SOLO has been successfully tested according to the standards DIN EN 12182 and DIN EN 12184 at the Technical University of Berlin, Berlin-Cert.

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Table of contents

1	Symbols and safety instructions.	04
2	Description of the product	08
3	Mounting the attendant's control	09
3.1	Initial Operation	10
3.2	Driving with the SOLO	12
3.3	Transfer to the control unit	14
3.4	Turn off the SOLO	15
4	Warning notices, malfunction and trouble shooting.	15

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1 Symbols and safety instructions

WARNING - Can lead to serious injuries or death!

CAUTION - Can lead to light injuries!

NOTE - Can lead to material damage!

- » Before initial operation, read the user's manual carefully and take note of the warning and safety instructions as well as the wheelchair's limiting values.
- » Before initial operation, check the electronic function, anti-tilt supports, brackets and, if applicable, the powered wheels (7 bar) and the steering rollers' air pressure.
- » Before initial operation, store the SOLO at least one hour in a non-freezing environment.
- » Take note of the technical data.
- » The approved total weight may not be exceeded.
- » Due to the mounting of the SOLO the measurements of the wheelchair can change.
- » Operation only by one trained person. This person should be physically and intellectually capable to handle the SOLO safely.
- » Practice outside on level ground and at the lowest possible speed.
- » For the transportation of persons, not of loads.
- » Secure persons who can not seat on their own with a belt.

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

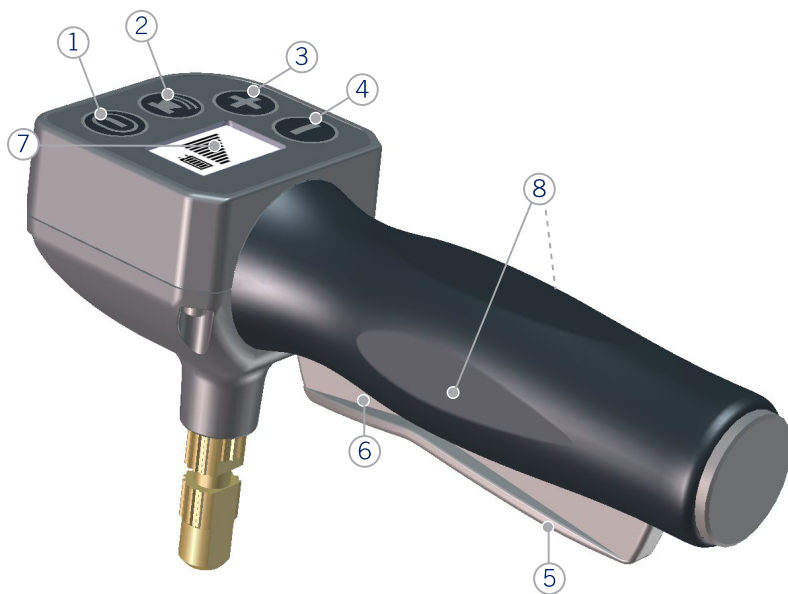
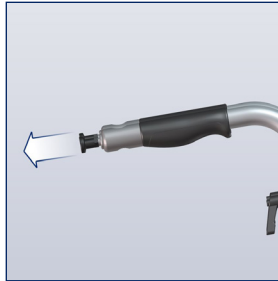
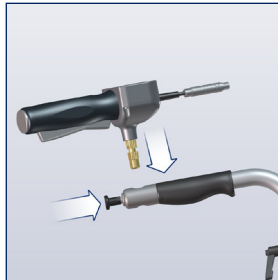
Put the grip into the back grip adaptor and make sure the locking bolt catches.
» Control grip on the right!

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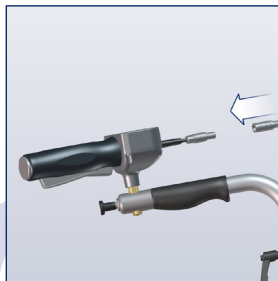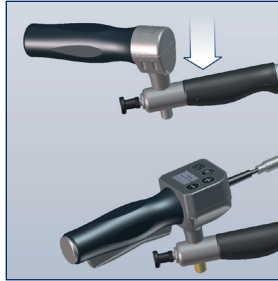Repeat the process and attach push bar on the left.

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Standard display control unit
 » Battery capacity
 » Speed selection

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Confirm the transfer to the control unit with “+” or “-“.

- » Press “+” key to permit the transfer to the attendant’s control.
- » Press “-“ to suppress the transfer to the attendant’s control.

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Display control unit after transfer.

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WARNING - Never touch rotating or moving parts!

CAUTION - If the drive elements block or the driving characteristics change, turn off the SOLO immediately and stop using it!

CAUTION - When driving uphill the control unit reacts more sensitively. Please briefly let go of the control grip when the deflective movements are too strong.

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Note the chapter on warnings, error indications and trouble shooting in the original user's manual!

The display's light turns itself off after 20 seconds of non-use.

During longer brakes of non-use, the automatic switch-off turns the SOLO off after 10 minutes.

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Check battery capacity.

- » 5 bars = full
- » 3 bars = half full
- » 1 bar = empty -> charge!

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Move attendant control in the direction you want to go.

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Press rocker switch in the direction you want to go.

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**Panic switch**

Press panic switch all the way -> SOLO goes into freewheel. When released, the SOLO brakes immediately.
 » This is independent of the travel direction.

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**3.3 Transfer to the control unit**

Press the ON/OFF key briefly (< 2 seconds).

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Confirm the transfer to the control unit with “+” or “-”.

- » Press the “+” to permit the transfer to the control unit!
- » Press the “-” key to suppress the transfer to the control unit!

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4 Warning notices, malfunction and trouble shooting

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Please refer to the original manual of the SOLO.

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