# PROTEKTOR

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Thank you for purchasing PROTEKTOR the umbrella for rollators and for your trust in ROLKO products. Before using this product the first time, carefully read the entire user manual. Check that all parts are included and follow all of the safety instructions.

## 01. Application and purpose

The rollator umbrella PROTEKTOR offers an ideal protection against rain and sun. The rollator umbrella base ensures fast and easy installation to standard 4-wheel rollators and also wheelchairs, allowing you to push your rollator or use your wheelchair safely with both hands. The base leaves enough space for shopping bags, cane holders, back belts, etc.

## 02. Contraindication

Our umbrella is not suitable for all applications that deviate from the intended use and/or may lead to danger. The rollator umbrella is not to be used, if no suitable attachment point is available on the wheelchair/rollator, the supplied plastic clamps of the umbrella base do not properly fit the wheelchair/rollator tube or the tipping safety is no longer guaranteed. Please have this checked by your retailer and take note of the safety instructions below. This product is also not suitable for Delta walkers (risk of tip-over!)


## 03. Safety Instructions

- Before using the umbrella the first time, make yourself familiar with the operation of the umbrella and the attachment. When using the PROTEKTOR, please take its height and size into consideration.
- The instructions for use are part of the umbrella and must be handed over when the umbrella is passed on.
- Open the umbrella (with the automatic) never close to your body. To avoid injury, make sure, no other person is close around. Consider the space requirement of the open umbrella.
- Check before every use if umbrella base and umbrella are firmly attached.
- The fixation unit must be installed in such a way that the braking function of the rollator is not impaired when the umbrella is opened or closed. Lighting, bell, cane holder, etc. must not be affected. Please also note the assembly instructions under 4: operation and fastening.
- Attach the umbrella-base in such a way to rollator that you can easily close and fold the umbrella. Shut the umbrella and tie the Velcro straps, before folding it together.
- Attention: Danger to pinch the fingers while closing umbrella or adjusting the joint-connectors!
- Always watch for an unobstructed view, in particular in road traffic. Consider height and size of the open umbrella.
- If the wind conditions are too strong, the rollator umbrella must be closed, not suitable for Delta walkers (tip-over danger!)
- Do not move, nor carry the rollator by the umbrella base.
- The tilting stability of the rollator can change negatively after attaching the umbrella. Follow the installation instructions and get advice from a specialist dealer.

## 04. Operation and Attachment


### Operation

Bring the umbrella into the desired position, by pushing the spring-loaded button (3) on each of the joint connectors. In addition, the umbrella can be rotated the by loosening the Star screw of the umbrella mounting-clamp. Subsequently firmly tighten the Star screw again! Open the Velcro straps (1) of the umbrella and push the release button (2). Attention: the umbrella opens immediately. If not in use and to save space, the umbrella can be folded away on the joints.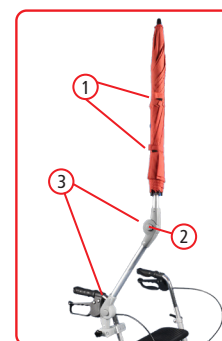


## Attachment

The umbrella base is already prepared for oval tube attachment, at the right side of the rollator (fig. a). For other tube shapes, replace clamps and Allen screws as needed.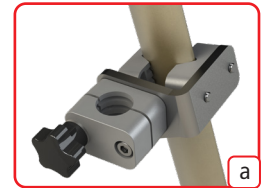


*Oval tube assembly*


- 1 Choose the matching clamps: oval tubes (base setup, fig. a), round tubes (18/20/22/25 mm, fig. b), or mix-match clamps for custom tubes (fig. d).
- 2 Choose the appropriate Allen screw (35/45/60/75 mm, fig. d) and attach the umbrella base via Allen wrench in desired position to the height adjustable vertical handle of the rollator.
- 3 Adjust the tilt angle of the umbrella base to the incline of the handle-tube, by first removing the Star screw completely. After adjustment, insert the Star screw back again into hole I or II of the bracket (fig. g+h).
- 4 To attach the umbrella to a horizontal tube, first remove Allen screw and Star screw. Thereafter attach the umbrella base again to the bracket via holes I and IV (fig. c+f).
- 5 Insert the umbrella with the shaft into the umbrella base and firmly tighten the Star screw. If required, re-tighten the Allen screw, to ensure a firm fit of the Umbrella with the base!
- 6 For an installation on the left side of the rollator turn the umbrella base, adjust the tilt angle of the umbrella base to the incline of the handle-tube, by first removing the star screw completely. After adjustment insert the Star screw back again into hole III or IV of the bracket (fig. e).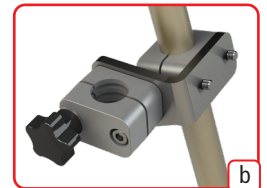


*Round tube assembly*

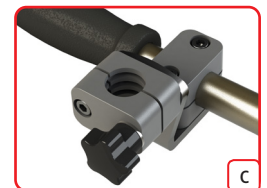
*Horizontal assembly*

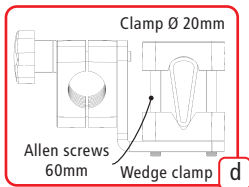
*Combination of clamps for custom-tube assembly*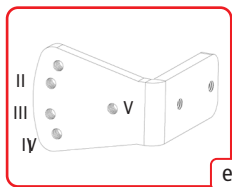


*Hole pattern bracket*

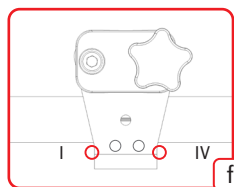*Push handle attachment*

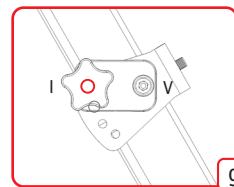*Attachment to an inclined tube*

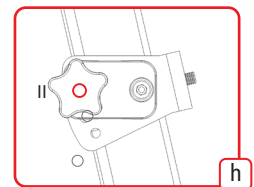*Attachment to a vertical tube*

## 05. Cleaning and Disinfecting

Wash off the umbrella with a damp cloth, if needed with the addition of a mild cleaner. Use lukewarm water and, if necessary, non-abrasive, pH-neutralized, commercial household cleaners. Always allow the board to dry completely. If necessary, the umbrella can be disinfected with household disinfectants (spray or wipe disinfection). Always follow the application instructions of the disinfectant manufacturer. To dry off the umbrella fabric, the umbrella must be stored spanned. Chlorine, organic or anorganic substances with active chlorine may not be used, because they change material properties, which may cause material damages and for this reason hazards to the user or helper.

## 06. Maintenance

Check the umbrella and its attachment base for visual defects before every use. Make sure that the material is undamaged and that the umbrella is in proper condition. Check once a month at least:

- the locking function and rotation of the articulated joints,
- the automatic opening of the umbrella,
- the tightness of the mounting unit on the device and the condition of the plastic clamps,
- the condition of the umbrella tip and fabric.

## 07. Re-use

The product is suitable for re-use. The effects of weather and UV radiation, mechanical stress as well as natural aging or careless handling may cause a premature wear, over which the manufacturer has no direct influence. If the umbrella fabric is damaged, the automatic opening shows malfunctions, the joints are unstable or defective or the fixation unit is defective, the umbrella must be disposed of. Tube clamps for the fixation unit can be reordered at specialist shops. For information regarding cleaning and disinfection, please refer to section 5.

## 08. Disposal


All parts of the rollator umbrella may be safely disposed of as a household waste. To do this, disassemble the rollator umbrella into its individual parts as far as possible to enable the materials to be sorted afterwards.

## 09. Technical Data

Item no. (without reflector stripes): HA-SCH-1-BG (brown green), HA-SCH-1-GG (yellow gray), HA-SCH-1-R (red)  
Item no. (with reflector stripes): HA-SCH-1-BG-REF (brown green), HA-SCH-1-GG-REF (yellow gray), HA-SCH-1-R-REF (red)  
Weight: 1,1 kg  
Umbrella Ø (opened): 105 cm  
Suitable for tubes with Ø 18 - 25 mm and oval tubes up to 60 mm  
Materials: Polyester, aluminium, polyamide, steel, polypropylene, ABS.

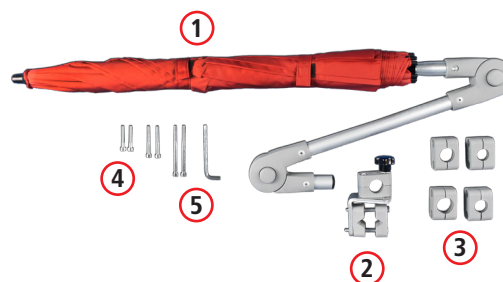
### Delivery Content

1. Rollator umbrella with joint connectors
2. Umbrella base, pre-assembled with wedge clamps and 2 Allen screws M6x60
3. Mounting clamps for tube Ø 18/20/22/25 mm
4. Set of Allen screws M6x35/M6x45/M6x75
5. Allen wrench
6. User manual

### Label

-  manufacturer
-  consider operating instructions
-  item no.
-  production date / batch

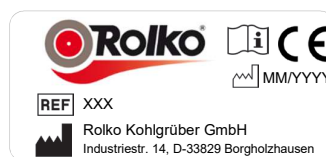As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN 12182. The EU declaration of conformity is available for download at [instructions.rolko.com](http://instructions.rolko.com).

**Operating Instructions: Rollator umbrella PROTEKTOR / 2020-07-03 / Rev. 14**  
**HA-SCH-1-BG(-WZ), HA-SCH-1-GG(-WZ), HA-SCH-1-R(-WZ), HA-SCH-1-BG-REF(-WZ), HA-SCH-1-GG-REF(-WZ), HA-SCH-1-R-REF(-WZ)**  
**Basis-UDI-DI: 426067605PROTEKTOR00000XR**

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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