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Badewannensitz

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GB Instructions for Use

Bath seat

With back rest 9403-B and Without back rest 9403-A

These instructions for use apply for the Order Numbers:
000224 - 000225 with Pad and 411224 - 411225 without Pad.

Reutäckerstrasse 12
D-76307 Karlsbad-Ittersbach
Tel. +49 (0) 7248/ 9186 - 0
Fax. +49 (0) 7248/ 9186 - 86

Table of Contents Bath Seat

General	3
1. Preliminary remark	4
1.1. Symbols used and their meaning	4
1.2. Scope and Condition of Delivery	4
1.3. Transport / Packaging / Unpacking / Checking	5
1.3.1. Unpacking	5
1.4. Notes of the Manufacturer for the Specialist Trade	5
1.5. Directives, Laws, Norms	5
1.6. Labeling and type plates	6
2. Appropriate Use and Indication	6
2.1 Appropriate use	6
2.2 Indication	6
2.3 Counter-indication	6
3. Safety	7
3.1. Overall Risk Assessment	7
3.2. General Notes on Safety	7
4. Components	7
5. Use	8
5.1 Adjusting bath seat	8
5.2. Positioning the bath seat	8
5.3. Sitting	9
6. Cleaning	9
7. Malfunctions / Troubleshooting	9
8. Maintenance / Inspection	9
9. Technical Data	9
10. Disposal	10
11. Guarantee	10

General

Compliance with EC Directive 93/42 for medical products is confirmed for the bath seat described in the following.

Device No.:.....

These instructions for use are part of the scope of delivery.
They must be kept in a safe place by the user and remain with the product if re-sold.

We reserve the right to alterations due to technical developments compared to the description provided in these instructions for use.
Reproduction, translations and copies in any form, including extracts, require written approval from the manufacturer.
The manufacturer owns copyright.
These instructions for use are not subject to any amendment service. You can find out the current status from:

1. Preliminary Remark

Dear User,

Thank you for choosing this technically perfect, reliable product designed in line with state of the art technology.

Please read the instructions for use **before** assembling and using the bath seat. These instructions for use apply for bath seats with and without back rests.

These instructions for use are intended for all users, especially care staff, as a basis for safe and hazard-free use of the bath seat.

These instructions describe the preparation, operation, use, maintenance and care required for the product.

To gain maximum use of the bath seat and to maintain handling safety, please read the instructions for use, observe the respective instructions for use and ensure that the instructions are always accessible for the user.

1.1. Symbols used and their meaning

WARNING!

Indicates a potentially dangerous situation. If you do not avoid such a situation, this may result in personal or material damage.

IMPORTANT!

Indicates very useful information in specific contexts and notes which must be observed.

1.2. Scope and Condition of Delivery

The scope of delivery includes:

- 1 bath seat
- 1 Instructions for use
- 1 packaging and transport box (can be re-used)

1.3. Transport / Packaging / Unpacking / Checking

The bath seat is supplied ready to use, has been set up by a specialist or care staff and the functions have been checked.

Check the packaging before unpacking. If the packaging is damaged, the contents may also be damaged.

If damage to the contents can be identified, notify the carrier immediately.

Damage to the Device!

If you drop the device, there is a risk of damage and impairment of function.

1.3.1. Unpacking

The bath seat is completely packed in foil. After unpacking, dispose of the foil in accordance with local disposal regulations.

Do not burn the foil! Toxic gases may develop when burning.

After unpacking, check the contents for completeness in accordance with the delivery documents.

1.4. Notes of the Manufacturer for the Specialist Trade

Specialized trade checks the walking frame for

- completeness,
- for damaged parts, especially that no parts are bent,
- whether the delivery can generally be assembled easily, and all parts function,
- adjust the bath seat to the bath pursuant to Ch. 5.1.

1.5. Directives, Laws, Norms

Directives, laws, norms and regulations applied:

- Directive of the Council 93/42 EEC, Medical Products
- MPG, (Medical Products Act) of the Federal Republic of Germany

1.6. Labeling and type plates


Type and manufacturer plate on the underside of the stool:

Sign plates must be clearly legible. Illegible or missing signs must be replaced immediately.

The type plate is very important for the identification of the product.

The type plate must not be removed.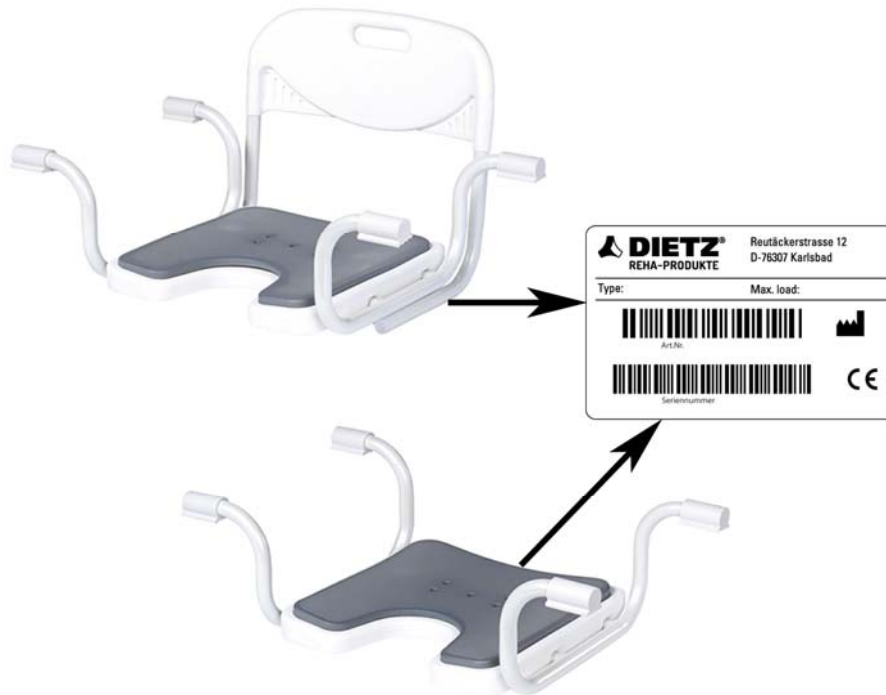

2. Appropriate Use and Indication

2.1 Appropriate Use

The bath seat is exclusively for use by persons up to a max. body weight of 120 kg, for safe body care in the bath.

2.2 Indication

Use of the bath seat is appropriate in the case of

- Inability to stand
- Insecurity in standing
- Reduction in or lack of functionality of lower extremities

2.3 Counter-indication

The shower chair must not be used by persons who

- Cannot control their balance when sitting

3. Safety

3.1. Overall Risk Assessment

If used properly and in accordance with its intended purpose, it is not dangerous to use the bath seat.

The materials used and the construction comply with EC directives, the Medical Products Act and the norms for medical products.

3.2. General Notes on Safety

Risk of slipping!

When sitting down and getting up from the bath.


Inform persons may slip when getting in or out of the bath. The care person must check the stability of the bath seat.

Risk of jamming!

When setting the width of the bath seat and when reaching into the gap between frame and seat, you must not jam your fingers.

4. Components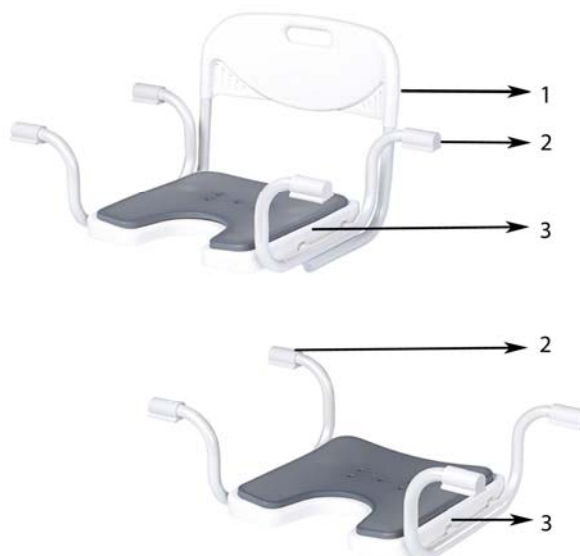

Back rest **(1)**


Bath edge rest **(2)**

Seat **(3)**

5. Use

5.1 Adjusting the bath seat

The bath seat has to be adjusted to the bath tub width. The adjustments on the underside of the seat are for this purpose.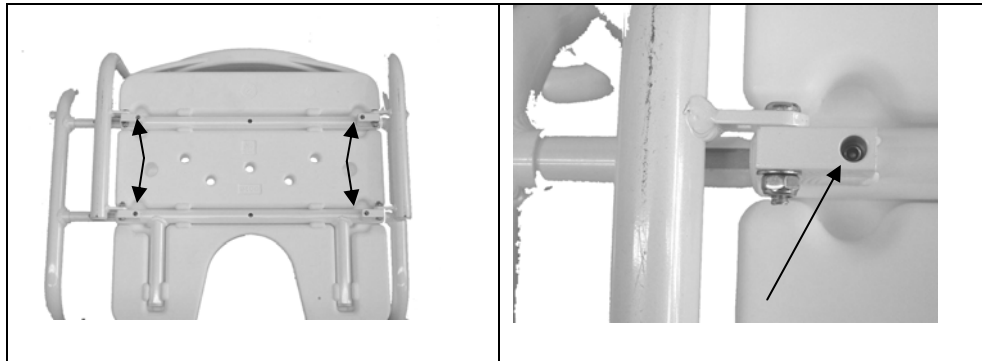

Symbol photo, subject to technical alterations


- ⇒ Unscrew the 4 Allen Key bolts with an Allen key SW 4.
- ⇒ Place the bath seat in the bath and pull out the two sides of the resting frame so that the resting rubbers are positioned completely on the bath edge.
- ⇒ Make sure that both sides are pulled out equally.
- ⇒ Now remove the bath seat from the bath and screw the 4 screws tight.

5.2 Positioning the bath seat

The bath seat can be used in every conventional bath tub with normal dimensions. The resting brackets have a non-slip cover (1). This prevents slipping of the unit on the bath edge.

Place the bath seat in the bath so that the resting brackets are completely positioned on the bath edge.

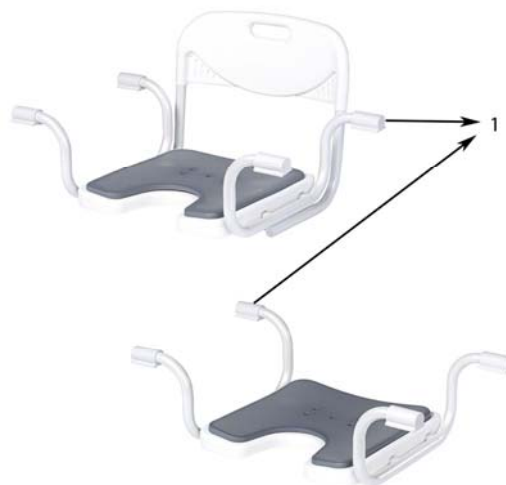5.3 Sitting

Risk of slipping!

Always ask a care person to help you to get into the bath and use the seat.

Hold tight to the bath when getting in and out.

Risk of tilting!

If one or several of the anti-slip plastic covers is damaged or is missing, the seat may slip. In such a case, do not use the seat.

6. Cleaning

Clean with conventional disinfectant.

- Wipe the bath seat with a damp cloth and
- Disinfect occasionally using conventional cleaning agents

7. Malfunctions / Troubleshooting

Give the bath seat to a specialist trader if malfunctions are to be repaired.

8. Maintenance / Inspection

This bath seat is maintenance-free. If servicing is required, please take the bath seat to a specialist trader

9. Technical Data

Order No.	Total width cm	Total length cm	Total height cm	Seat cm	Weight kg	Max. user weight kg
000224 411224	73,5-89	54,5	40	40 x 40	3,6	120
000225 411225	73,5-89	54,5	25,5	40 x 40	2,7	120

10. Disposal

When the bath seat has reached its service life limit or cannot be used for other reasons, the components are to be disposed of by the specialist trade.

11. Guarantee

Guarantees cover all defects of the product verifiably caused by material or manufacturing defects.

The guarantee period is 24 months as of readiness for shipment, and at the latest after delivery.

The guarantee does not cover the following:

Damage caused by natural wear and tear,

by intent or by negligent, inappropriate operation or use.

The same applies if unsuitable cleaning agents, lubricants or fats are used.

The guarantee certificate must be completely filled in when submitting a complaint.

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