D/GB

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Translation of the original user manual

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CE-Declaration of conformity

The company AAT Alber Antriebstechnik GmbH herewith declares that the v-max⁺ is conform to the current regulations of the European Union, EEC Guidelines 2006/42/EG, 2007/47/EG and 93/42/EG.

This declaration becomes void, if the v-max⁺ has been altered without the consent of AAT Alber Antriebstechnik GmbH.

Signature of the manufacturer:

Markus Alber
(Managing shareholder)

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The v-max⁺ has been successfully tested according to the standards DIN EN 12182, DIN EN 12184 and the applicable parts of ISO 7176 at the Technical University of Berlin, Berlin-Cert.

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Table of content

1	Symbols and safety instructions.	07
2	Description of the product	11
2.1	Features of the device	11
2.2	Technical data	15
2.3	Dimensioned sketch	17
2.4	Scope of delivery	18
2.5	Accessories	19
3	Assembly	20
3.1	Wheels with quick-release axle	20
3.2	Battery pack	21
3.3	Power unit	22
3.4	Attach the grips into standard support handles	23
3.5	Attach the grips in conjunction with a transverse pipe	24
3.6	Anti-tilt supports	25
3.7	Disassembly	26
4	Initial operation	27
4.1	Turn on the v-max+	27
5	Operation	28
5.1	Driving the v-max+	30
5.2	Transport.	32
6	Care and maintenance	33
6.1	Batteries	33
6.2	Charger	40
6.3	Cleaning/Desinfection	43
6.4	Recycling.	44

6.5	Safety checks45
6.6	New owner45
7	Warning notices, malfunction and trouble shooting	46
7.1	Warning notices46
7.2	Trouble shooting48
8	Warranty and liability.	49
8.1	Warranty49
8.2	Liability50

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1 Symbols and safety instructions

WARNING - Can lead to serious injuries or death!

CAUTION - Can lead to light injuries!

NOTE - Can lead to material damage!

- » Before initial operation, read the user manual and take note of the warning and safety instructions as well as the wheelchair's limiting values.
- » Before initial operation, check the electronic function, the anti-tilt supports, brackets and, if applicable, the steering rollers' air pressure.
- » Before initial operation, store the v-max+ at least one hour in a non-freezing environment.
- » Take note of the technical data.
- » The approved total weight may not be exceeded.
- » Due to the mounting of the v-max+ the adjustment of the seat/backrest can be restrained and the measurements of the wheelchair can change.
- » Operation only by one trained person. This person must be physically and mentally able to handle the v-max+ safely and must not suffer from cognitive impairment.
- » If possible, always operate with both hands.
- » Practice outside on level ground and at the lowest possible speed.

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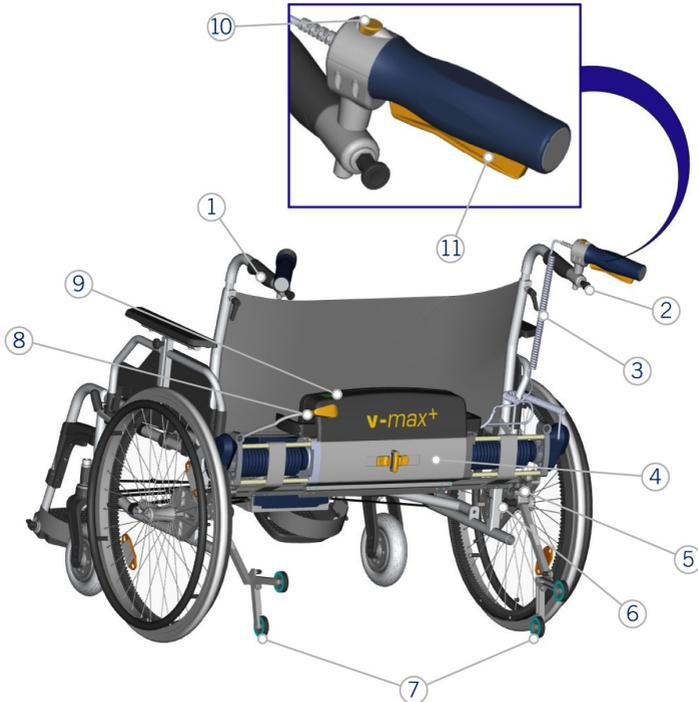
2.2 Technical data

Technology	v-max ⁺
Measurements	Power unit: Height = 262 mm Width = min. 600 mm - max. 1010 mm Depth = 167 mm
Weight	Power unit incl. grip and cables = 13.5 kg Mega battery pack = 14.3 kg Power cable = 0.2 kg Battery bag plus velcro straps = 0.3 kg Battery charger = 0.3 kg Total weight (without wheelchair) = 28.6 kg
Total weight approved	300 kg ⁽¹⁾ (person, wheelchair, v-max ⁺)
Limiting values for mounting	min. seat width 60, maximum wheel camber 5%
Speed, forward/backward	5,5 km/h 3 km/h
Range with one charge of the batteries	up to 22 km ⁽²⁾
Braking - / climbing capacity	up to 12 % ⁽²⁾
Batteries	2 x 12 V/16 Ah
Direct current motor	24 V / 130 W
Noise level	The device's A weighted noise level lies typically below 66 dB (A)

Keys	Rocker switch, forward; 3 N Rocker switch, backward: 4 N Panic switch, forward: 40 N Panic switch, backward: 30 N Keys: On/Off, horn, speed pre-selection): 0,74 N
Storing/operating temperatur: device	-20° C up to +60° C / -10° C up to +50° C
Storing/operating temperatur: battery pack	-20° C up to +60° C / -10° C up to +50° C
Vibrations	The weighted RMS value of acceleration the upper body is exposed to lies typically below 2,5 m/s ² .
Class of application	A, B
Sticker serial number	Power unit

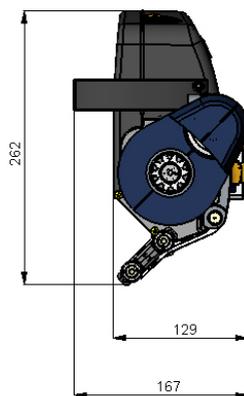
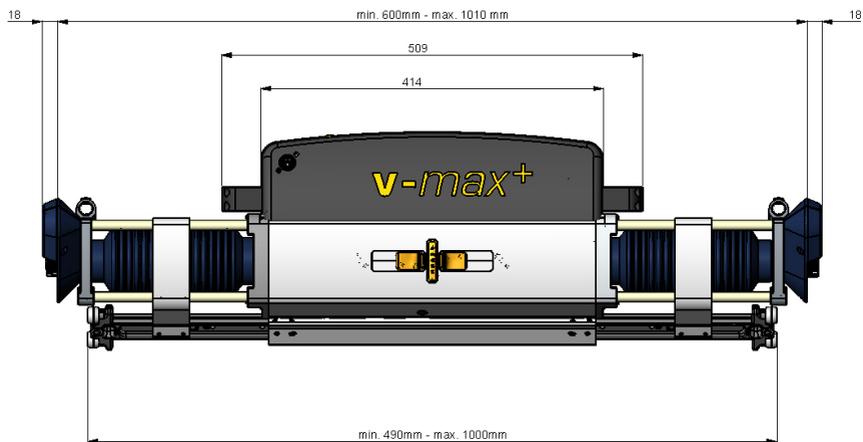
⁽¹⁾ Please observe the maximum load according to the wheelchair manufacturer's stipulations.

⁽²⁾ According to ISO 7176-4, the range depends on the person's weight, the underground, the territory, the temperature, etc.

In the interest of progress we reserve the right for technical changes.

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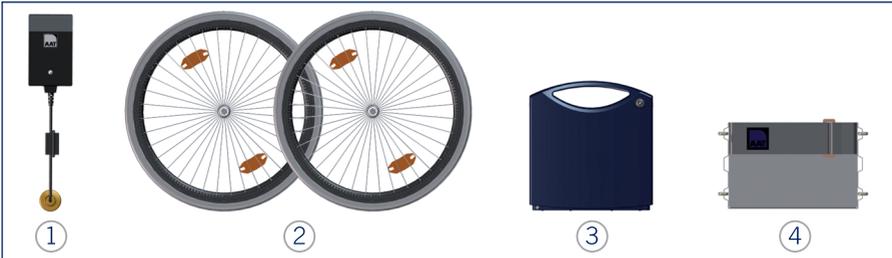
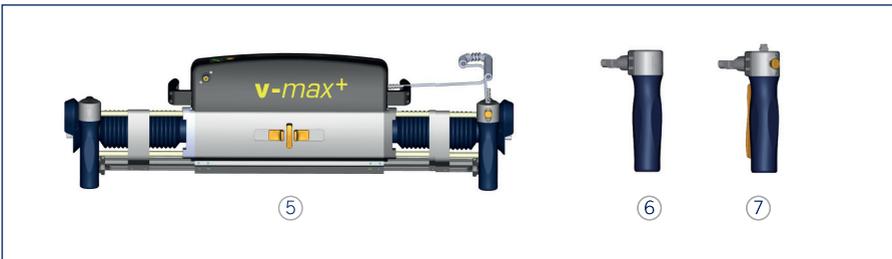
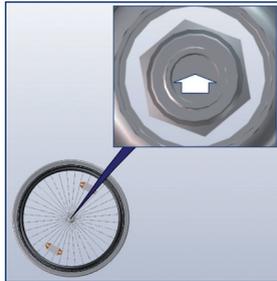
WARNING - The wheels with quick-release axles must lock!

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
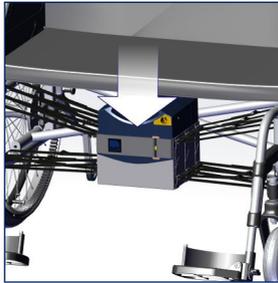
Insert into the bag and close the Velcro fastening.

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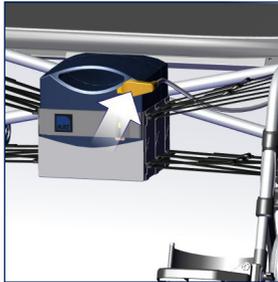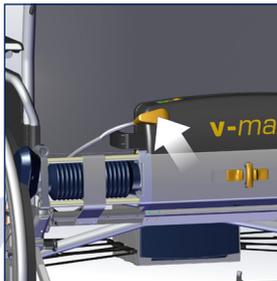
Insert into brackets all the way
» evenly on both sides!

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3.4 Attach the grips into standard support handles

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WARNING - Always tighten tightening knobs!

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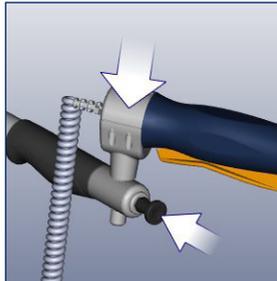
Put the grip into the back grip adaptor and make sure the locking bolt catches.

- » Control grip on the right!
- » Pushing grip on the left!

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If your wheelchair is equipped with a transverse pipe the grip adaptors must be mounted in advance.

» The mounting may only be done by the manufacturer or authorized personnel!

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Make sure to find a comfortable position of the grips.

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Put the grip into the back grip adaptor and make sure the locking bolt catches.

» Control grip on the right!
» Pushing grip on the left!

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WARNING - Never use the v-max+ without both anti-tilt supports!

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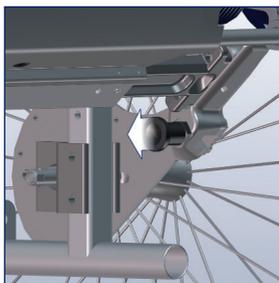
WARNING - Before initial operation check the anti-tilt supports!

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Should there be no anti-tilt support on your wheelchair – use AAT anti-tilt support in pairs!

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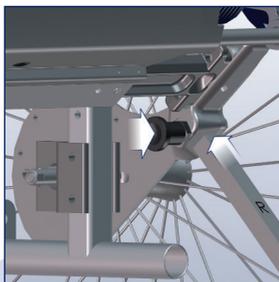
Slide into the bracket and make sure the locking bolt catches.

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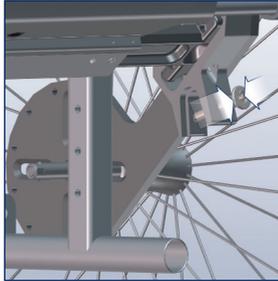
**Bracket with Quickpin**

Press the button and pull out the quickpin.

Slide into the bracket and secure it with the quickpin.
» Note identification “R” and “L”!

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Please repeat picture 1-2 on the other side.

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WARNING - The wheelchair's locking brakes need to be adjusted properly!

WARNING - Before initial operation, check electronic function, anti-tilt supports and brackets!

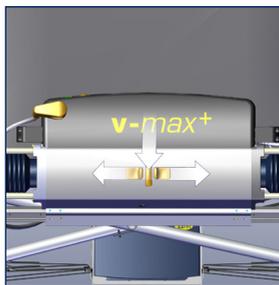
NOTE - Before initial operation, store the v-max+ at least one hour in a non-freezing environment!

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
4.1 Turn on the v-max⁺

- Use the PRESS button to move out the drive shafts.
- » Both sliders release.
- » The drive shafts extend.

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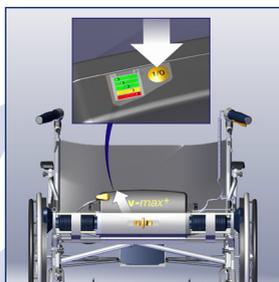WARNING - Operation only by one trained person! This person should be physically and intellectually capable of handling the v-max+ safely!

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WARNING - Practice outside on level ground and at lowest speed possible!

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WARNING - Never use the v-max+ without anti-tilt supports!

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WARNING - Use the lowest possible speed when driving on slopes!

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WARNING - Always go backwards over curbs/obstacles!

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WARNING - Do not drive parallel to steep mountain sites or tracks!

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WARNING - Keep a safe distance to curbs!

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WARNING - Do not drive too close to stairs or precipices!

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WARNING - When crossing streets, intersections and train tracks, use extra caution!

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WARNING - Avoid using the v-max+ in heavy rain, ice, snow and on unsecured grounds (e.g. sand, mud, loose gravel)!

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WARNING - Avoid driving down slopes manually!

WARNING - When not in use, e.g. while using mass transportation (bus, subway, etc.) or when you change seats, turn off the v-max+, unplug the power cable and engage the locking brakes on the wheelchair!

WARNING - Do not use the v-max+ on escalators, stairs or moving belts!

WARNING - Driving on public roads, observe traffic regulations!

WARNING - Always operate with both hands!

CAUTION - Never use grips to lift or push, instead use the wheelchair's pushing grips!

NOTE - Do not drive backwards to a wall with mounted anti-tilt supports!

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WARNING - Never touch rotating or moving parts!

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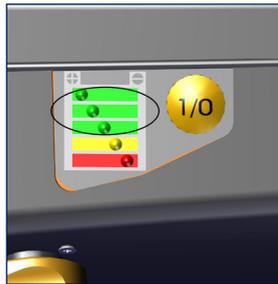
CAUTION - If the drive elements block or the driving characteristics change, turn off the v-max⁺ immediately and stop using it!

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
Note the chapter on warnings, error indications and trouble shooting!

Grün = voll

Green = full

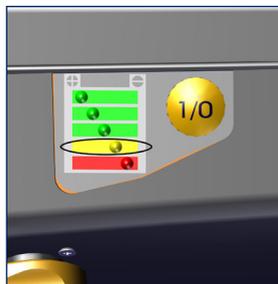Gelb = gering
» Akku laden!


Yellow = low
» Charge battery!
Red = empty
» Charge battery!
» Audio warning every 12 sec.!

30

Rot = leer

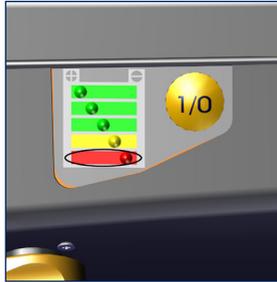
- » Akku laden!
- » Warnton alle 12 Sek.!

Red = empty!
Charge the battery!
Sound signal repeated
every 30 seconds!

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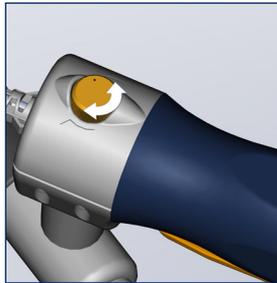Choose driving speed.
» Practice at the lowest possible speed.

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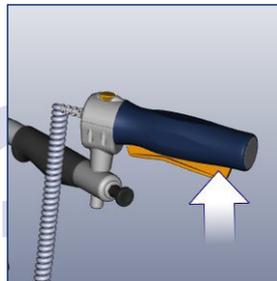
Press rocker switch in the direction you want to go.
» Steer with the grips!

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Panic switch

- Press panic switch all the way
- » v-max+ is free running
- » v-max brakes when releasing the rocker switch.
- » Independent of travel direction!

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WARNING - Do not use the v-max+ plus wheelchair as a seat in a vehicle!

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WARNING - During transportation in vehicles, secure the v-max+ according to relevant legal regulations!

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CAUTION - Turn off the v-max+ for transportation!

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CAUTION - Observe weight specifications!

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NOTE - Do not lay down the wheels with quick-release axle on their handrims!

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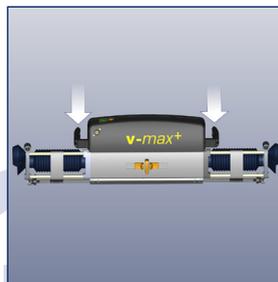
The battery pack is approved for air freight by DOT and IATA. However, inform airline ahead of time.

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6 Care and maintenance

WARNING - We recommend a safety check every two years! Inspection plate informs about the latest date for next inspection!
Tip: We recommend a safety check, if the v-max+ goes to a new user!

WARNING - Repairs and safety checks may only be done by the manufacturer or authorized personnel!

6.1 Batteries

WARNING - Never short-circuit or connect the battery pack's poles!

WARNING - Remove the charger/power cable before exchanging the fuses!

CAUTION - When charging the v-max+, make sure the cable lies on safe ground!

CAUTION - Never mend or bridge defect fuses!

NOTE - Protect the v-max+ against humidity!

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NOTE - Never carry the charger at the charging cable!

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NOTE - Charge the battery pack after each use!

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NOTE - Protect the charger and the voltage converter for the car from oils, grease, aggressive cleaning agents, thinners or damage!

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NOTE - Observe the specifications on the charger!

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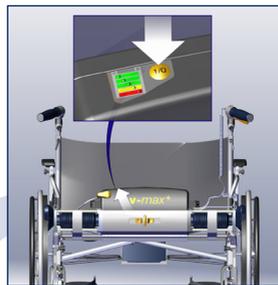
Battery pack is admitted to air travel by DOT and IATA (observe chapter 5.5).

The battery-pack is maintenance free and rechargeable.

It is not possible to use the v-max+ while it is charging.

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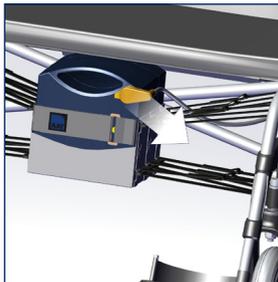
If desired:
Put the grips back into the adaptors on the device.

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
Move the two sliders in direction to the PRESS button.
» Both sliders engage.

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Unplug the power cable.

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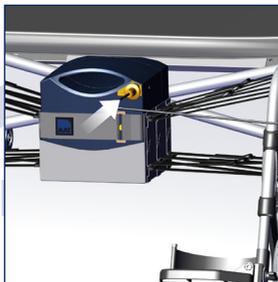Connect charger.

LED grün = Akku voll
» ca. 10 Stunden

LED green = batteries full
» approx. 10 h

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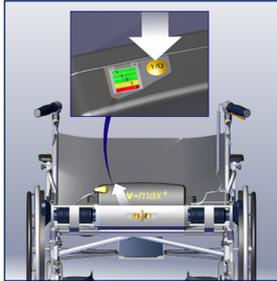
LED orange = batteries are
being charged

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
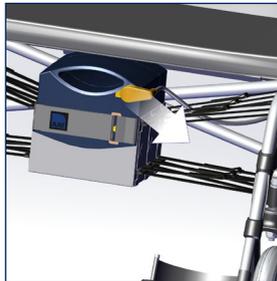
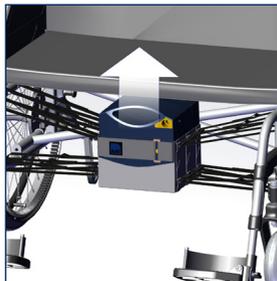
Unfasten Velcro fastener and remove battery pack.

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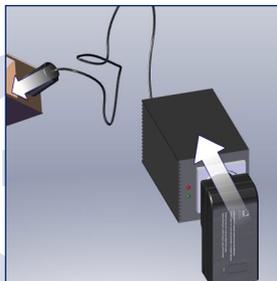
Connect the voltage converter for the car and the charger.

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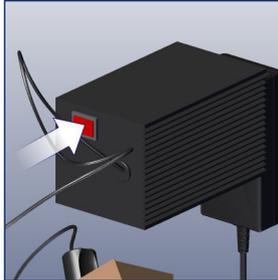» Warnton = Spannung gering!

Turn on voltage converter for the car.

» Acoustic signal = low voltage!

LED grün = Akku voll
» ca. 10 Stunden

LED green = batteries full
» approx. 10 h

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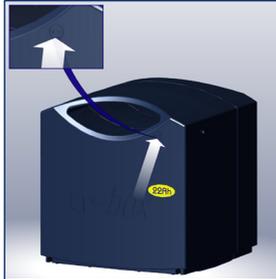
LED orange = batteries are being charged

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Change fuse.

» Upper fuse = spare fuse!
» Spare fuses available at
the manufacturer.

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Push in fuse box.

» Make sure the locking
mechanism catches.

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NOTE - The charger may only be used in dry rooms!

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NOTE - Do not use in potentially explosive rooms (garages)!

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NOTE - If the charging device is brought from a cold room into a warm room, condensation can form. Wait using the charger in this case until the condensate has evaporated!

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NOTE - Only charge rechargeable sealed batteries!

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NOTE - The charger is not to be used by small children.

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NOTE - Only charge the battery in a well ventilated area!

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NOTE - Disconnect the power cable when not in use or during cleaning and maintenance!

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NOTE - Do not bend the cable and the plug or expose to any pressure!

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NOTE - Never pull on the power cable to remove the charger from the socket.

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NOTE - Never carry the charger on the power cable or the power supply!

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NOTE - Do not use the charger when it is defective!

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NOTE - Never subject the charger to humidity (water, rain water, snow) during the charge process!

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NOTE - Never use the charger if it has incurred a hard blow, was dropped or damaged otherwise. Bring the damaged charger for repair to a authorized personnel!

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NOTE - Take the output data (voltage and current) from the inscriptions on the charger!

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NOTE - Only use the supplied AAT-charger to charge the battery pack!

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NOTE - The charger may only be operated at 100 V - 240 V AC!

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NOTE - Do not touch the plug with wet hands!

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NOTE - Keep the v-max⁺ away from ignition sources!

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NOTE - Toothed segments and driveshafts must be cleaned of dirt and deposits.

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NOTE - Clean the wheelchair according to the manufacturer's specifications!

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NOTE - Do not use a high-pressure cleaning device!

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With a moist cloth and mild household cleaners / disinfectants.

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6.4 Recycling

The v-max⁺ and its battery pack are products of long durability.

However, if your v-max⁺ is old and has reached the end of its lifespan you may return its components to AAT Alber Antriebstechnik GmbH or authorized personnel to be recycled.

A new law concerning electric and electronic devices (ElektroG) has been passed on March 24, 2005 regulating the taking back and recycling of old electric and electronic devices.

According to the Federal Ministry for the Environment (German BMU) the v-max⁺ is considered a means of transportation and does not fall under this law concerning the recycling of old electric or electronic devices (ElektroG, §2 paragraph 1 sentence 1 (cross reference, Guideline of the European Parliament 2002/96/EG)).

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WARNING - Safety checks may only be done by the manufacturer or authorized personnel!

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A safety check is mandatory before the device changes hands. To conduct a safety check it is necessary to attend a training course. Please check the homepage for further information.

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The v-max+ is a device suitable for multiple use. Before the v-max+ changes hands it must be cleaned, disinfected and pass a safety check (see chapter 6.2).

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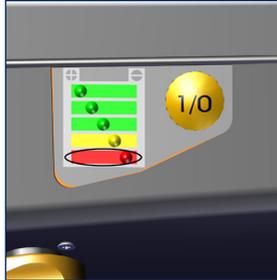
7 Warning notices, malfunction and trouble shooting

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Malfunction

Red LED blinks	Type of Malfunction
1x	Malfunction of the motor, the motor controller or the battery pack
2x	Low voltage, batteries empty
3x	Overvoltage
4x	Electro-magnetic brake at fault
5x	Indicator for excess voltage
6x	Indicator for overstraining
7x	Temperature controller

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7.2 Trouble shooting

Problem	Check and potential solution
v-max+ cannot be turned on	Is the power cable plugged in the power unit and in the battery pack? No? Plug it in.
	Is the battery pack empty? Yes? Charge it.
	Is the fuse defective? Yes? Change it.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
v-max+ does not drive	Is the v-max+ still turned on? No? Turn it on.
	Immobiliser activated? Yes? deactivate!
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
Battery pack doesn't charge	Is the charger connected? No? Connect it properly.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
Fuse often damaged?	Yes? Contact the manufacturer or authorized personnel!

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8 Warranty and liability

8.1 Warranty

AAT Alber Antriebstechnik GmbH grants a warranty of two years for the v-max⁺ including all its accessory parts (with the exception of the battery pack). The warranty begins at the date of purchase and covers failure of material and errors in manufacturing.

For the battery pack, always assuming correct maintenance AAT Alber Antriebstechnik GmbH grants a warranty of six months from the date of purchase.

The warranty becomes void concerning defects or damage which are the result of the following:

- » damage due to excessive strain.
- » damage due to improper handling.
- » violent damage.
- » natural wear and tear (driving belt, toothed segments, wheelchair wheels, drifeshafths, keypad).
- » improper and insufficient maintenance.
- » unauthorized constructional changes or inappropriate use of the v-max⁺ or its accessories.
- » Using the v-max⁺ exceeding the approved total weight.
- » improper charging of the battery pack.

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8.2 Liability

The manufacturer AAT Alber Antriebstechnik GmbH, is neither liable nor responsible for potential damage of the v-max+ in case of:

- » improper handling of the v-max+.
- » repairs, fittings or other works performed on the v-max+ by unauthorized personnel.
- » operation not in accordance with the user manual.
- » extraneous parts added or connected to the v-max+.
- » parts removed from the v-max+.
- » operation exceeding the approved total weight.
- » no regular biannual maintenance check performed by AAT Alber Antriebstechnik GmbH or its authorized personnel.
- » operation of the v-max+ with insufficiently charged batteries.

Important safety advice for the US market!

Important EMI-Information:

- » 20 Volts per meter (V/m) is a generally achievable and useful immunity level against interference from radio wave sources (as of May 1994) (the higher the level, the greater the protection).
- » This device has an immunity level of 20 V/m with no accessories connected to the device.
- » Nevertheless, always check whether the device's electronic functions work properly before each use. This is particularly important, if there is an aerial mast nearby.
- » Please note that the v-max+ may interfere with other electronic devices' sensitive electro-magnetic fields (e.g. anti-theft systems in department stores). This may also happen, if you connect other devices.
- » Also make sure that the v-max+ is not operated in close proximity to medical devices with a high potential for danger and/or life-supporting functions as well as diagnostic devices.

D

GB

Notizen:

Notes:

52

D

GB

Notizen:

Notes:

53

D

GB

Notizen:

Notes:

54

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