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1 General instructions

1.1 Introduction

These operating instructions contain information on how to use the transfer aid safely and properly.

Read the operating instructions thoroughly prior to the use of the transfer aid and follow all instructions.

Keep all documents in a safe place.

1.2 Proper use

The transfer aid is used on a bathlift and utilised to rotate persons with reduced mobility into bathing position.

In its version as transfer and rotating aid the transfer aid is additionally used to transfer persons with a more reduced mobility from the rim of the bathtub to the bathlift.

Any other form of use is not permitted.

The transfer aid may not be used by children.

Contraindications

No contraindications are known if the product is used properly.

1.3 Disposal

The transfer aid consists of recyclable materials.

Old appliances and packaging must be disposed of and recycled in accordance with the valid national regulations.

1.4 CE mark

The product complies with the medical products directive (EU) 2017/745.

1.5 Reuse

The product is suitable for reuse. The amount of reuse depends on the condition of the product in question.

Prior to reuse, follow the manufacturer's reconditioning and hygiene guidelines, and check the technical and functional safety of the product as well.

1.6 Service life

The product is designed for a service life of at least 3 years, provided that it is used in accordance with its intended use and that all cleaning and safety instructions are observed.

This service life can be exceeded if the product is carefully handled, cared for, and used.

2 Safety instructions

Do not overload the transfer aid (max. user weight: 140 kg).

Always put the weight on the centre of the rotary seat of the transfer aid.

Hot water, sunlight or hot air can cause the surface of the product to become hot. Risk of burns!

Check the transfer aid for functionality prior to every use. Do not use a damaged transfer aid.

In case of severely reduced mobility, use the transfer aid with the help of an assistant.

Serious incidents relating to the product must be reported to the manufacturer and the competent authority.

3 Description

The transfer aid is available in the following versions:

- Transfer and rotating aid (Fig. 1A)
- Rotating aid (Fig. 1B)

The transfer aid consists of a rotary seat (Fig. 1A/1B) with seat cover (Fig. 5) and a base (Fig. 2A/2B). The rotary seat is rotatably connected to the base. In the case of the transfer and rotating aid, the rotary seat can be shifted, too.

The base has several moveable links which adapt to the bathlift such that the transfer aid does not tilt. Rubber pads on the moveable links prevent the transfer aid from slipping.

3.1 Technical data

Refer to "Technical data" at the end of the operating instructions.

3.2 Label

The label (Fig. 4) is attached to the bottom side of the transfer aid and contains the following information (exemplary illustration):

Sales or service address (1, Fig. 4), conformity marking (2, Fig. 4), note (observe operating instructions) (3, Fig. 4), maximum user weight (4, Fig. 4), period of production (5, Fig. 4), product identification number (6, Fig. 4), product name (7, Fig. 4), marking as a medical product (8, Fig. 4), batch number (9, Fig. 4), product type (10, Fig. 4), barcode (11, Fig. 4), item number (12, Fig. 4) and manufacturer (13, Fig. 4).

4 Use

CAUTION Do not allow any oily or strongly soapy substances to get to the rubber pads. Otherwise, the transfer aid may slip from the bathlift.

CAUTION Do not reach underneath the rotary seat while rotating. Finger can be crushed.

CAUTION When using as transfer board, keep a still position and avoid excessive body movements. Otherwise, there is a risk of slipping.

4.1 Transfer and rotating aid

1. Raise bathlift to entry height.
2. Place transfer aid on seat section such that the rotary seat in the end position is in the centre of the seat section (1, Fig. 3A).
3. Push rotary seat towards the rim of the bathtub (2, Fig. 3A).
4. Sit down on the rotary seat.
5. With the rotary seat, move to the centre of the seat section (3, Fig. 3A).
6. Lift legs and swivel into the bathtub (4, Fig. 3A).
7. Lower bathlift.

To get off: First, slide to the rim of the bathtub with the rotary seat and, then, swivel legs out of the bathtub.

4.2 Rotating aid

1. Raise bathlift to entry height.
2. Place transfer aid in the centre of the seat section (1, Fig. 3B).
3. Sit down on the rotary seat.
4. Lift legs and swivel into the bathtub (2, Fig. 3B).
5. Lower bathlift.

5 Maintenance and care

5.1 Care

NOTICE Do not use any scouring agent. Otherwise, the plastic surfaces may be damaged.

You can clean the product with a dishwashing or laundry detergent and lukewarm water. Keep the plastic pieces clean using an off-the-shelf plastic cleaner (without solvents and with a pH value of 5-9).

The seat cover can be removed and washed at 40 °C in a washing machine.

If necessary, or when using the product for more than one person, the surface can be disinfected with an off-the-shelf disinfectant (max. 70 % alcohol) **after cleaning**. Depending on the disinfectant used, the disinfectant will be sprayed on or applied with a cloth. Follow the disinfectant or detergent manufacturer's instructions.

When adhering to the specified cleaning and safety instructions, the product is maintenance-free.

After cleaning, dry off the transfer aid with a clean cloth.

5.2 Removing the seat cover

1. Remove plug in the centre of the rotary seat (1, Fig. 5).
2. Pull seat cover slightly back at the rim until you can remove it (2, Fig. 5).

5.3 Attaching the seat cover

1. Place and align seat cover on the rotary seat.
2. Push edges of the seat cover into the recesses of the rotary seat (3, Fig. 5).
3. Fasten seat cover with the plug.

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