

Instructions for Use

Booster Toilet Seat

TSE-1



**DIETZ**<sup>®</sup>  
REHA-PRODUKTE

Reutäckerstraße 12  
D-76307 Karlsbad-Ittersbach  
Tel. +49 (0) 7248/ 9186 - 0  
Fax. +49 (0) 7248/ 9186 - 86



# Booster Toilet Seat TSE-1

## Contents

<b>General Information</b> .....	<b>3</b>
Preliminary Remarks .....	3
Compliance declaration .....	3
Appropriate Use.....	3
Indication .....	4
Signs and Symbols.....	4
Liability.....	4
<b>Notes on Safety</b> .....	<b>4</b>
<b>Overview of Product and Delivery</b> .....	<b>5</b>
Inspection of Delivery .....	5
Equipment of the Booster Toilet Seat TSE-1 .....	5
Labeling and type plates .....	6
<b>Assembly</b> .....	<b>6</b>
Assembly of arm-rests .....	6
Setting the seat height .....	7
Position of booster toilet seat .....	7
Attaching the booster toilet seat .....	8
<b>Use</b> .....	<b>9</b>
<b>Notes on Care for the User</b> .....	<b>10</b>
Cleaning .....	10
Inspection of the booster toilet seat by the user.....	10
<b>Malfunctions/Troubleshooting</b> .....	<b>10</b>
<b>Disposal</b> .....	<b>10</b>
<b>Technical Data</b> .....	<b>11</b>
<b>Guarantee</b> .....	<b>12</b>

## **General Information**

### **Preliminary Remarks**

Thank you for choosing this booster toilet seat from our company.

The booster toilet seat TSE-1 can be mounted quickly and easily to almost every standard WC basin. It enables the raising of the seat position by up to 16 cm.

Top-quality design which can be adjusted in height ensures an ergonomically formed seat with maximum comfort. The arm-rests provide additional support when sitting down and getting up.

Please read these instructions for use carefully before using the booster seat. It contains important notes on safety and valuable tips for use and care. If you have any questions or require additional information, please contact the specialist trader who delivered your booster toilet seat.

These instructions for use contain all important notes on adjusting and operating the TSE-1.

### **Compliance declaration**

Dietz GmbH declares sole responsibility for compliance of the booster toilet seat TSE-1 pursuant to EC Directive 93/42 for medical products.

### **Appropriate use**

The booster toilet seat TSE-1 is suitable for toilet use by persons up to a maximum body weight of 120 kg.

The adjusted height of the booster toilet seat and the movable arm-rests make it much easier to shift from the wheelchair to the toilet.

The booster seat is not suitable as a seat.

Please ask your specialist trader to explain the functions and operation of the seat.

## Indication

Toilet assistance is appropriate and required for persons with

- Lack of control of their torso
- Imbalance disorders
- Loss of function in lower extremities
- Persons with stiff hips and spine
- Persons with no hands, quadriplegics or persons with disabled arms and considerable disorder of gripping capacity
- Signs and symbols

	Warning! Indicates very important safety-related information. Observe operating instructions!
	Important! Indicates very useful information for the respective context.

## Liability

Dietz GmbH only accepts liability if the products are used under the conditions specified and for the purpose specified. We recommend that products be handled properly and in accordance with the instructions for use. Dietz GmbH is not liable for any damages caused by components and spare parts which have not been released by Dietz GmbH. Repairs must be carried out by authorized specialists or by the manufacturer himself.

## Notes on Safety

The booster toilet seat is no risk if used for the appropriate purpose.

The maximum user weight is 120 kg.

The lid of the booster toilet seat is not suitable as use as a seat, only sit on the toilet seat.

Clothing or parts of the body may become jammed when sitting down and getting up.

Clothing and parts of the body may also become jammed in the brackets when flapping down the arm-rests.

Make sure the arm-rests are locked in correctly.

Always fasten the bars at the back and front at the same height.

Make sure all bolts are tight.


## Overview of Product and Delivery

### Inspection of delivery

The booster toilet seat is delivered packed in a box. After unpacking, please store the box if possible. It serves for subsequent storage of the booster toilet seat. After receiving the product, please check for completeness and perfect condition. The contents comprise:

- 1 packaging and transport box
- 1 pre-assembled booster toilet seat
- 2 arm-rests
- 1 instructions for use

### Equipment of the Booster toilet seat TSE-1

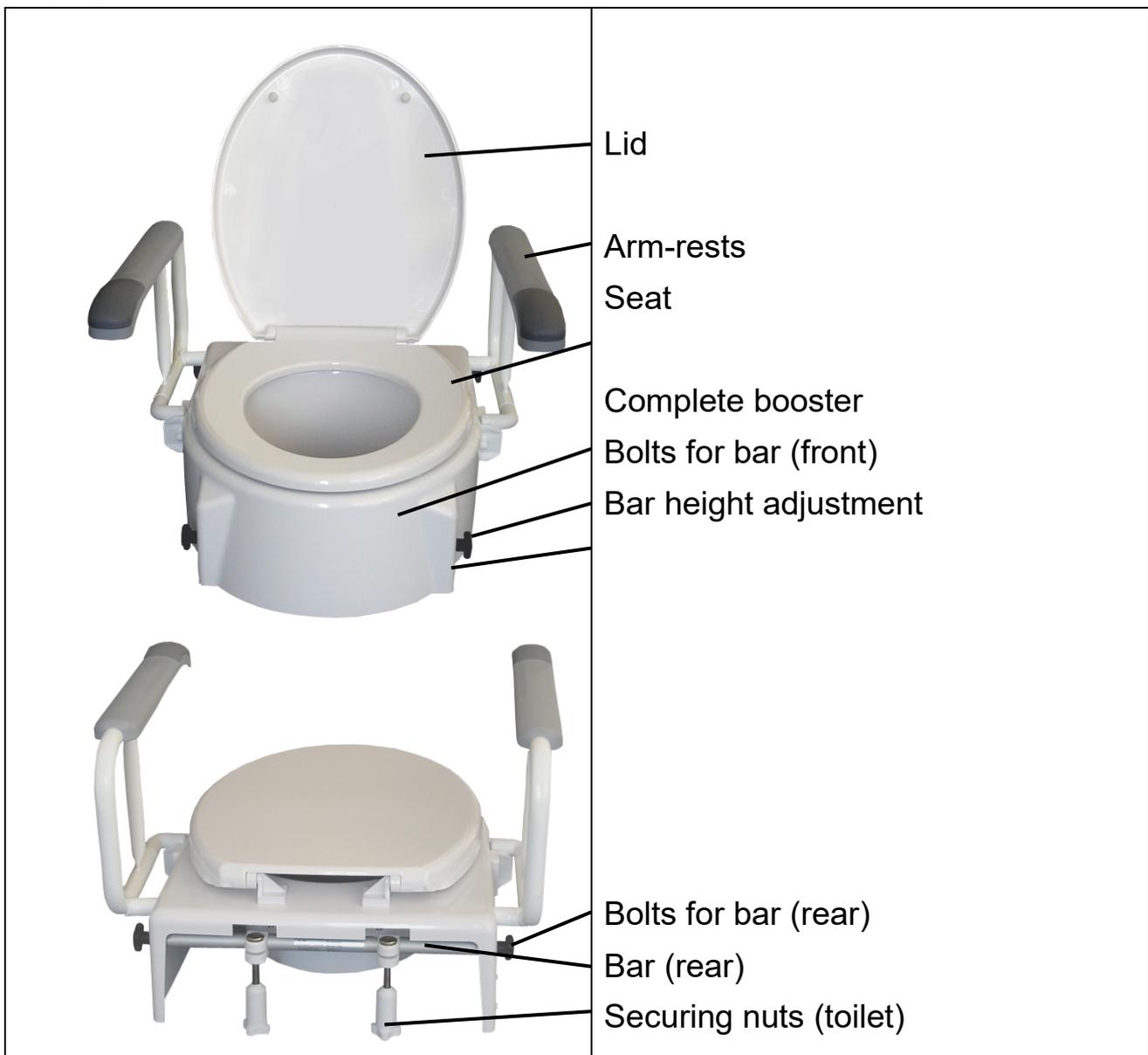

## Labeling and type plates

Type plates and manufacturer plates are attached to the frame under the seat.



Plates must be legible. Illegible or missing plates must be replaced immediately. The type plate is important for identification of the product. It must not be removed.

## Assembly



The booster toilet seat can be mounted to every conventional toilet bowl.



The distance from the toilet to the rear wall must be at least 18 cm to be able to flap down the arm-rests.




Before mounting the booster toilet seat, check that the toilet bowl is correctly fitted.

## Assembling the arm-rests

Press down safety button ①.

Then push the fastening tube of the arm-rest into the bracket of the booster toilet seat ②.

Make sure the safety button catches after you have mounted the arm-rests.

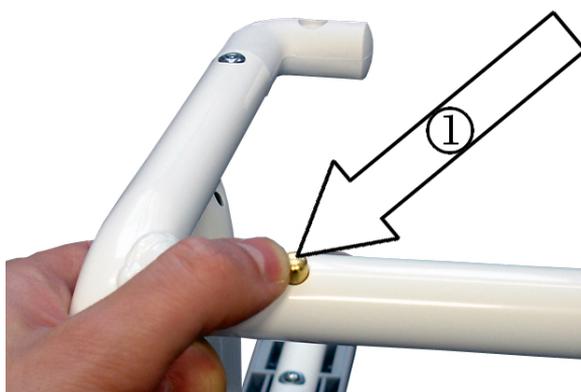

## Setting the Seat Height


The seating height of the booster toilet seat can be easily adjusted to your requirements.

First, remove the hole covers ① by pushing them out from the inside.

Unscrew the twist buttons ② to remove the bearing bar.

Move the bearing bar to the desired height ③ and re-tighten.

Now re-apply the hole covers.

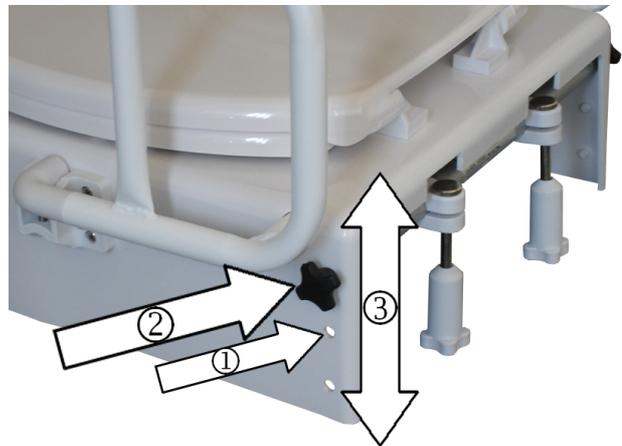


Repeat this procedure with the other bearing bar and fasten the bearing bar at the same height.

## Position of the Booster Toilet Seat

The position of the booster toilet seat can be easily adjusted to the front and to the back.

To do this, you have to install the rear bearing bar twisted by 180°.

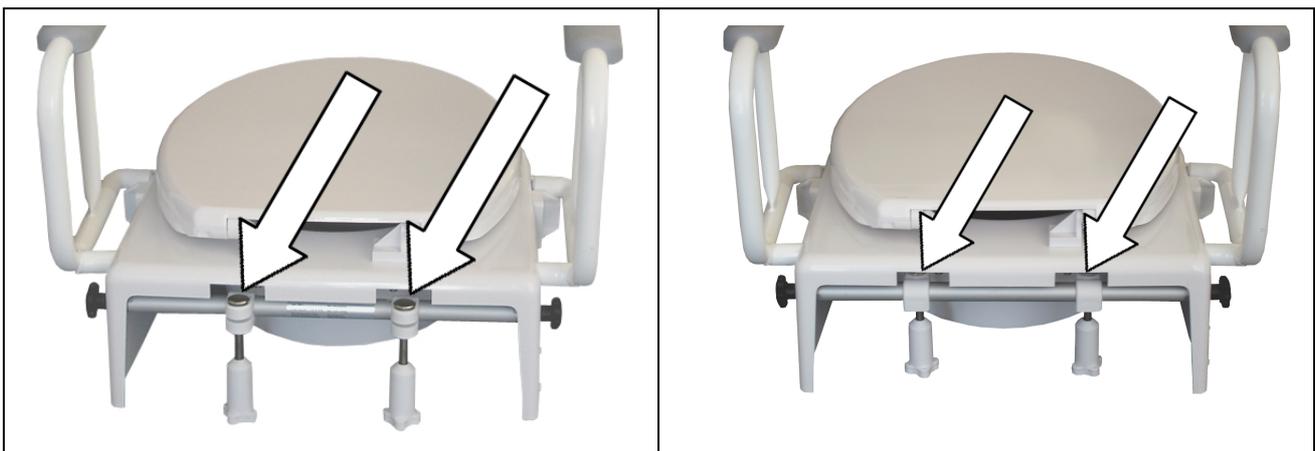

## Booster Toilet Seat TSE-1

### Attaching the booster toilet seat

To mount the booster toilet seat, the old toilet seat first has to be removed.

Check if the factory-set height of the booster toilet seat meets your requirements. To do this, place the booster toilet seat on the toilet bowl. You can adjust the height.

Make sure you only mount the booster toilet seat as high as you need. Your feet should be on the ground at all times.

<p>The toilet seat height can be adjusted applying just a few hand movements.</p> <p>First, unscrew the plastic nuts from the thread rods.</p> <p>Push the thread rods through the holes in the toilet bowl and screw on the nuts gently at first.</p>	
<p>Now position the booster toilet seat in the center above the toilet bowl by moving the booster to the left or right using the rear brackets.</p>	



The booster must be tightly screwed on. Otherwise the booster may shift when you sit on it.



Check secure mounting of the booster toilet seat very carefully.

Dismantling of the booster toilet seat is carried out in reverse sequence.

## Use

Use the arm-rests for careful sitting down and getting up from the front.

When entering from the side (e.g. wheelchair), swivel the arm-rests to the back on the respective side of entrance.

Make sure that the arm-rests lock correctly when flapping them down.



Clothing or parts of the body may become jammed in the catch for the arm-rests when flapping down the arm-rests.



If possible, always sit up straight on the booster toilet seat. Beware of the increased risk of slanted mounting.

## **Notes on Care for the User**

### **Cleaning**

Normal cleaning and disinfection agents can be used to clean the booster toilet seat. Wipe the booster toilet set with a damp cloth.

Do not use any sharp or caustic cleaning agents or agents with grinding additives.

### **Inspection of the booster toilet seat by the user**

This booster toilet seat is maintenance-free. If servicing is required, please hand the booster to a specialist trader.

Inspection: Check tightness of all screws, bolts and nuts monthly and tighten if required.

### **Malfunctions/Troubleshooting**

Return the booster to a specialist trader in the case of troubleshooting and error rectification or in case of damage.

This does not include malfunctions which can be rectified by cleaning.



The specialist trader is to be supplied with necessary information and documents for repair and servicing, on request.

Dietz Customer Service is available to deal with any questions, on +49 (0) 7248/ 9186 - 0.

### **Disposal**

If your booster toilet seat or the packaging is no longer to be used and is to be disposed of, please contact your local disposal company to find out about respective regulations for disposal.

**Technical Data**

Order Number	000119
Width cm	54
Height cm	36
Length cm	50
Increase in seating position cm	8/ 12/ 16
Seat width cm	36
Seat depth cm	41
Opening cm	28 x 23
Distance between the armrests cm	43
Empty weight kg	4.7
Max. load capacity kg	120

# Booster Toilet Seat TSE-1

## Guarantee

Guarantees cover all defects of the product verifiably caused by material or manufacturing defects. The guarantee period is 24 months as of readiness for shipment, and at the latest after delivery. The guarantee does not cover damage caused by natural wear and tear, by intent or by negligent, inappropriate operation or use. The same applies if unsuitable cleaning agents, lubricants or fats are used. The guarantee certificate must be completely filled in when submitting a complaint (see below).



### **Guarantee Section**

Product:

Model name

Serial number

Specialist Trader:

Date and Stamp

Send to:

**DIETZ**<sup>®</sup>  
REHA-PRODUKTE

Reutäckerstraße 12  
D-76307 Karlsbad-Ittersbach  
Germany