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Preparing the operating condition	28
Using your wheelchair	
Getting in and out	
Swinging aside the side parts	
Folding up the foot plates	
Swinging aside and demounting the leg supports	
Moving ahead with the wheelchair	32
Pushing and slowing down with the help of an assistant	
Tip assists	
Independent moving and applying the brakes	
Locking brake	
Transport and folding	34
Adjustments	35
Locking brake	
Lower leg length	
Advice on product care	36
Cleaning	
Transfer/further use	
Checking the wheelchair by user	
Service notes for specialist retailers	37
Disposal	37
Technical data	38
Warranty	38
Service record	39

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# Users manual TOKIN 24“ wheelchair

## General information

Thank you for having chosen a Tomtar wheelchair. The design, functionality and quality of these modern wheelchairs won't disappoint you. Before operating the unit for the very first time, please read this manual carefully.

You will obtain vital safety instructions and valuable advice on the right use and maintenance of the wheelchair. If you have questions or require additional information, please contact your specialist retailer that supplied the wheelchair.

This users manual contains all information required to adjust and operate the TOKIN wheelchair.

## Declaration of conformity

Dietz-Reha Produkte GmbH declares under its sole responsibility that the TOKIN wheelchair described here conforms to the EU directive 93/42 for medical products.

## Purpose

The TOKIN wheelchair can be used both indoors and outdoors. The product is only to be used for self-transportation and for transporting people.

The wheelchair is designed for a maximum load of 110 kg. You can verify this information on the Serial number label.

## Indication

Inability to walk or severe walking disability as a result of

- paralysis,
- loss of limbs,
- meromelia/limb deformation,
- joint contractures/damage to joints (not on both arms),
- other ailments.

## Contraindication

Use of the wheelchair is not suited for

- perception disorders, severe disturbances of equilibrium,
- loss of limbs on both arms,
- joint contractures/damage to joints on both arms,
- inability to sit,
- reduced or insufficient eyesight.

## Signs and symbols

### Caution!

Indicates information that is particularly relevant to safety. Follow the instructions for use.

### Important!

Indicates information that is particularly useful in the respective context.

## Liability

DIETZ Reha-Produkte GmbH only assumes liability when products are used in accordance with the specified conditions and for the aforementioned purposes. We recommend using the products properly and maintaining them in compliance with these instructions. DIETZ GmbH Reha-Produkte does not assume liability for damage caused by components and spare parts not approved by DIETZ Reha-Produkte GmbH. Repairs may only be carried out by authorised specialist retailers or by the manufacturer.

## Safety notes

In order to avoid misuse and dangerous situations, you should practice safe handling of your new wheelchair on a flat and familiar surface. In such case, an assistant is highly recommended.

- Never leave children and youngsters unattended in the wheelchair.
- Prior to using the wheelchair, please check whether all parts attached have been fastened properly.
- The first attempts at moving the wheelchair should take place on a level and manageable surface. Familiarise yourself intensively with the braking and accelerating behaviour of the wheelchair when moving straight ahead and turning corners.
- Please note that padded parts can become hot in direct sunlight and lead to skin injury on contact. Therefore, cover these parts or protect the wheelchair from direct sunlight.
- The wheelchair may not be misused for transporting several persons or loads. Observe the specific function of the wheelchair.
- Never use your wheelchair while under the influence of alcohol or other substances that may influence your ability to react or your physical and mental receptiveness.
- Please note that redistribution of weight due to body movements or loading the wheelchair can increase the risk of tipping over.
- By all means, use ramps to avoid obstacles.
- Cross grooves, tracks and similar impassibilities in a right angle (90°), if you cannot move around them.
- There is a danger of tipping over to the rear starting at an angle of +10°.
- Avoid wheeling against obstacles (steps, kerbs) at speed or jumping down steps.
- Never stand on the foot plates to get in and out.
- Observe the road traffic regulations when on the road.
- If possible, wear bright, noticeable clothes. You will then be seen better by other traffic participants.

## Product and delivery overview


### Inspection of delivery

The wheelchair is completely packed in a cardboard box. Please keep the packing, if possible, it may serve for storing the wheelchair at a later time.

Upon receipt, immediately check the content for completeness and sound condition. The content consists of:

- 1 covering box
- 1 wheelchair preassembled
- 1 pair of leg supports
- 1 users manual

## Features of the TOKIN wheelchair

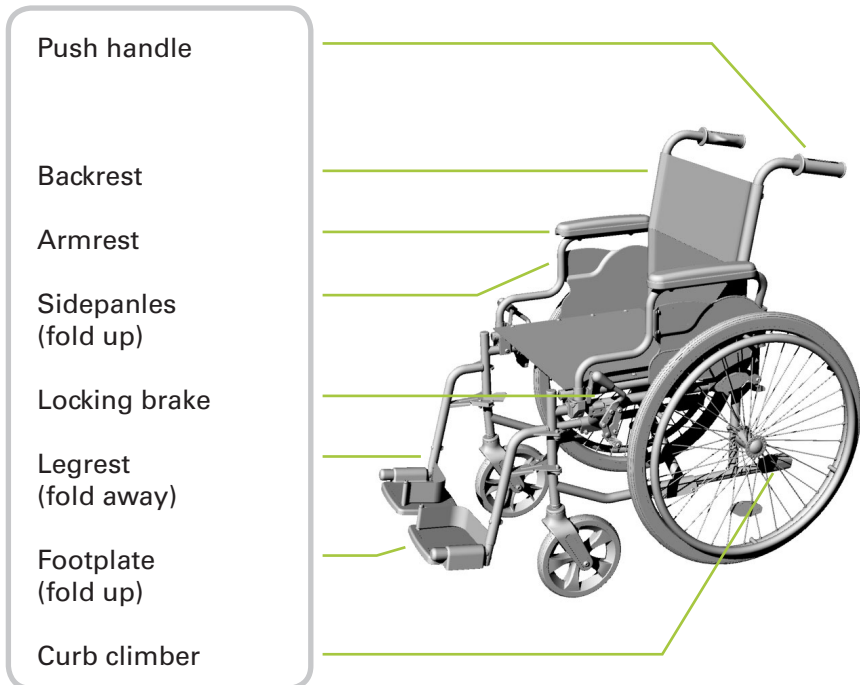## Labels

Serial number and safety labels are placed on the cross-bar under the seat.

### Caution!

All labels must remain readable. Immediately replace unreadable or missing labels. The serial number label is very important for identifying the product. It may not be removed.


## Preparing the operating condition

Generally, your wheelchair is delivered completely assembled, folded and with the drive wheels mounted. To make it ready for use, some easy and simple hand movements are required.


Position yourself behind the wheelchair and press down the cross-bar with both hands.

Make sure that the cross-bar engages properly with the supports.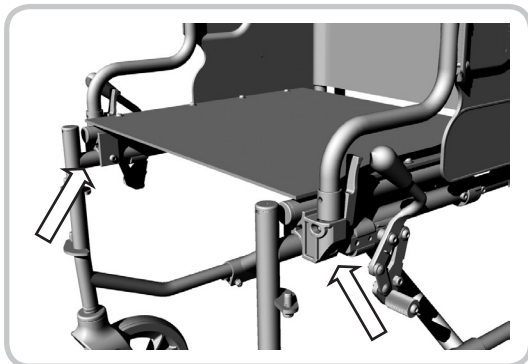

Mount the leg supports and swing them forward, they will snap in audibly.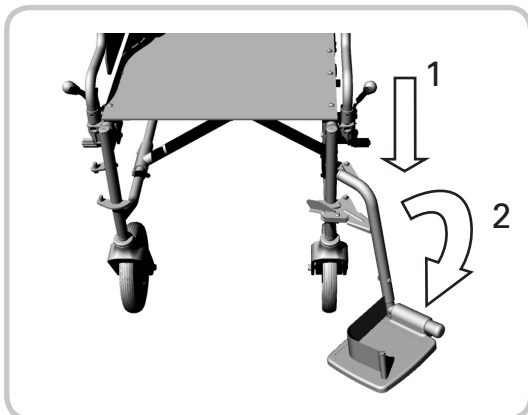

**Caution!**  
Danger of getting trapped! Watch your fingers when folding and setting up the wheelchair.

## Using your wheelchair

### Getting in and out sideways

The wheelchair allows you to easily get in and out. The foot plates, side parts and legrests can fold away or be retracted.

Place the wheelchair and the current or future seating surface as close beside one another as possible.

Should you wish to transfer from another wheelchair or another seating to the wheelchair, make sure that the wheels are secured by applying the locking brake.

Fold the sidepanel to the rear.

Fold the foot plates upwards.

Fold the leg supports towards the inside or exterior (if necessary, of both wheelchairs when you wish to transfer from one wheelchair to the other). Now slide sideways onto the seating surface.

Make sure you are sitting in the seat as far back as possible.

Finally, fold the sidepanel back down and the legrests back into their home position.

### Getting in and out forwards

Fold the foot plates upwards and pivot the legrests towards the exterior. (Description see above.)

Place the wheelchair and the current or future seating surface as close to the front of one another as possible.

Should you wish to transfer from another wheelchair or another seating to the wheelchair, make sure that the wheels are secured by applying the locking brakes.


Now slide onto the seating surface by twisting your body. Make sure you are sitting in the seat as far back as possible.

Subsequently, swivel the legrests back in centre position, fold down the foot plates and rest your feet thereon.

## Sidepanels

Press the locking lever (1) and lift the sidepanel (2) upwards.


When folding down, the armrests will automatically lock in place.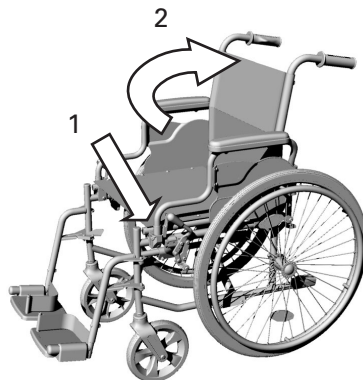

## Footplates

Fold up the foot plates to facilitate getting in and out.

To fold up the wheelchair in a space saving manner, the foot plates must be folded upwards.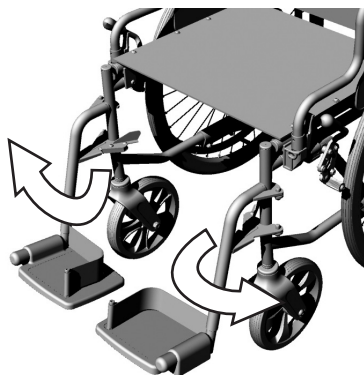


## Legrests

To remove the leg supports, first fold up the foot plates.

Now open the locking device and turn the legrests outwards.

Now you can remove the legrest in upward direction.

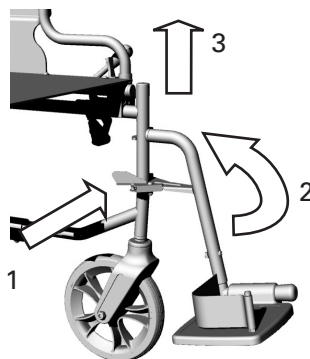## Moving ahead with the wheelchair

Pushing and slowing down with the help of an assistant

The assistant moves ahead the wheelchair by pushing the designated pushing handles.

Important!

The assistant must be physically and mentally capable of slowing down the wheelchair with own strength. The locking brake can be used as parking brake.

Tip assists

The tip assists allow the assistant to tip the wheelchair more easily and hence cross obstacles more easily.

Press down the tip assist with your foot. Balance can be maintained with the pushing handles.

## Independent moving and applying the brakes

Practice safe handling of your new wheelchair on a flat surface with the assistance of an aide, until you get accustomed to it. You may only participate in public traffic with sufficient travelling experience. Be especially careful when moving on inclined surfaces and when overcoming obstacles.

The wheelchair is set in motion and slowed down via the push rim.

To move the wheelchair ahead, put your hand on the push rim, whereas only your thumb and bent index finger touch the push rim.

The other fingers are closed to a fist.

To slow down, let the push rim glide between your thumb and bent index finger and slow it down with a tightening grip until the wheelchair comes to a stop.

To steer the wheelchair into a left and right direction, slow down the push rim on that side you would like to direct it.

To turn the wheelchair on the spot, one hand pushes forward, while at the same time the other hand pushes backwards.

**Caution!**

When moving the wheelchair, make sure that you do not touch the top of the tyre with your thumb.

Danger of getting jammed between tyre and sidepanel!

## Locking brake

Push the brake lever forward until it locks by itself.  
You can release the locking brake by retracting the lever.

## Transport and folding

For transportation in a passenger car or simply to clear space, your wheelchair can be folded up easily. It then only takes little space and can easily be stowed away.

Leave the wheelchair.  
Fold the foot plates upwards.  
Remove the legrests.

Pull the seat upholstery upwards in centre.  
The wheelchair will then fold up automatically.

## Adjustments

Only trained specialists should adjust brakes. However, should you have sufficient experience, you may also carry out this adjustment yourself.

Caution!  
Incorrectly adjusted brakes may result in life-endangering consequence


### Brakes

The distance between the brake bolts and the tyre cover must be ca. 17mm with the brake fully open.

To adjust the distance, loosen the bolts.

Move the brakes into the correct position.

Afterwards, tighten the bolts again and check the brake function.

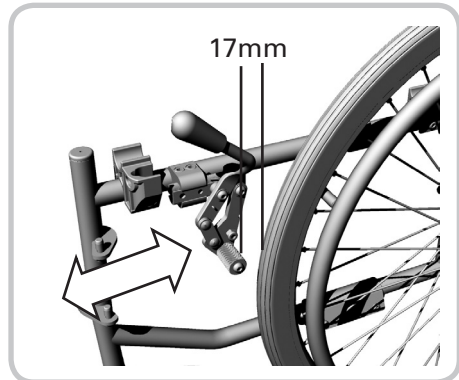### Lower leg length

Loosen the screw and move the foot plate into a position that is comfortable for you.

Tighten the screw again.

## Advice on product care

### Cleaning

- The frame parts of the wheelchair can be wiped using a damp cloth.
- In case of heavy soiling, use a mild cleaning agent. To clean the wheels use a damp brush with plastic bristles (do not use a steel brush!).
- The padding materials can be washed up to 40°C with mild soap solution and sanitised with disinfecting agents on an alcohol basis, where required.

Important!

Do not use a high-pressure or vapour cleaning device!

### Transfer/further use

The wheelchair can be reused. When the wheelchair is transferred reused by another person, ensure that all technical documentation necessary for its safe use is given to the new user.

The wheelchair must be cleaned, disinfected, examined by a specialist retailer and approved.

### Checking the wheelchair by the user

Prior to each use, please check:

- That the tyres are in good order and have sufficient air pressure (given optional pneumatic tyres). Check the profile depth of the tyres once a month. The profile depth should measure at least 1mm. Defective or worn tyres must be replaced.
- The operational reliability of the brakes. If the brake is in place, the wheelchair should be immovable. If the brake is released, the wheelchair should be easy to push without any grinding sounds. Should you notice any unfamiliar braking behaviour, immediately inform your specialist retailer and do not use the wheelchair any longer.

- The sound condition and cleanliness of the seat and backrest padding. If these are soiled, you can clean them in accordance with the instructions for cleaning. If they are damaged, contact your specialist retailer.
- All parts should work smoothly and not make noise. Squeaking noises indicate damage and should be examined with regard to their causes by a specialist retailer.
- That all screws and connections are tight.

## Service notes for specialist retailers

We recommend having your whole wheelchair inspected by a specialist retailer once a year. In case of faults and defects on your wheelchair, return it to your specialist retailer immediately for repair.

### Important!

On request, Tomtar will provide specialist retailers with required information and documents for repair and service. In case of queries, please contact the Tomtar Customer Service at phone No. +49 (0) 7248/ 9186 – 0.

### Important!

Use the enclosed service record to have an overview of the service status of your wheelchair.

## Disposal

If your wheelchair is not used any more and is to be disposed of, please enquire from local disposal companies about the applicable regulations.

## Technical data

Seat width cm		45	50
Seat height cm	51		
Seat depth cm	42		
Seat angle °	2		
Overall width cm		63.5	68.5
Overall length cm	107		
Overall height cm	91		
Back height cm	40		
Height of push handle cm	88		
Height of armrest cm	23		
Width of armrest cm	5		
Length of armrest cm	26		
Lower leg length cm	40 - 48		
Total weight kg	15		
Max. load capacity kg	110		
Drive wheels	24 x 1 3/8"		
Steering wheels Ø cm	20		
Tilting angle °	10		

### Caution!

The wheelchair is not approved as a seat in vehicles such as disabled person transportation vehicle.

## Warranty

This warranty covers all defects in the product that are shown to be caused by material or production defects. The warranty period is 24 months from receipt of the notification of readiness for despatch or from delivery at the latest. Our warranty obligations do not include damage caused by natural wear and tear, with intent, by negligent or improper operation or use. The same applies to the use of inappropriate care products, lubricants or grease. In the case of complaints, please contact your specialist retailer.

## Service record

Please use, if possible, the service codes provided on the bottom of this page for filling in the columns.

when what							
Reflectors lateral							
Reflectors rear							
Drum brake							
Parking brake							
Push handles							
Side panels							
Seat upholstery							
Back upholstery							
Steering wheels							
Drive wheels							
Tire pressure							
Tire							
Push rim							
Frame structure							
who							

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Dietz Reha-Produkte GmbH