


## Montageanleitung

Therapietisch für AS[01] / AS[01]RF


Art.-Nr.: 182231 / 182232

Der Therapietisch wird mit 3 Befestigungssets ausgeliefert

Set	Beschreibung
01	Halteschienen lang (400 mm) für Therapietisch zur Befestigung seitlich an der Armlehne
02	Halteschienen kurz (350 mm) für Therapietisch zur Befestigung unterhalb der Armlehne
03	Schraubenset für Therapietisch (4 Schrauben und 4 Kontermuttern)

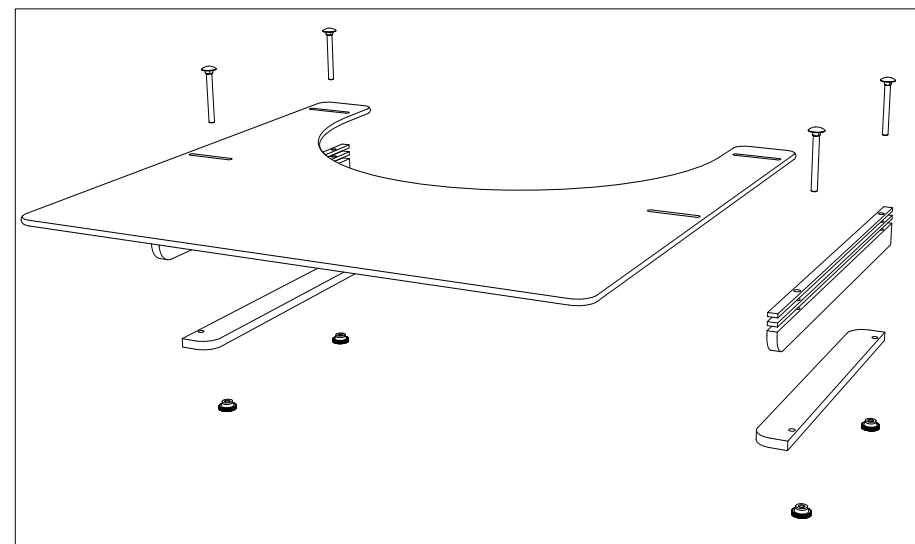

01)

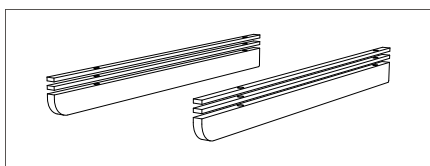


4 Schienen (L/B/H) á 400 x 10 x 4 mm  
2 Schienen (L/B/H) á 400 x 10 x 25 mm



Set 01) Je nach Rollstuhlmodell werden unterschiedlich viele Schienen verwendet :


AS[01] / AS[01]RF = pro Armlehne 1 x 25 mm Höhe

02)

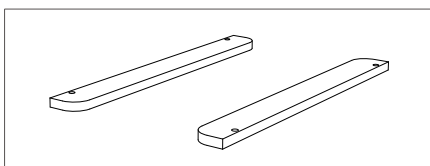

2 Schienen (L/B/H) á 350 x 38 x 10 mm

03)

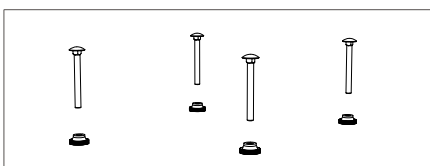

4 Schrauben & 4 Kontermuttern


## Assembly guide

Therapy table for AS[01] / AS[01]RF


Item no.: 182231 / 182232

The therapy table is delivered with 3 fixing sets

Set	Specification
01	Holding rails long (400 mm) for fixing the therapy table sideways at the armrest
02	Holding rails short (350 mm) for fixing the therapy table below the armrest
03	Screw set for therapy table (4 screws and 4 lock nuts)

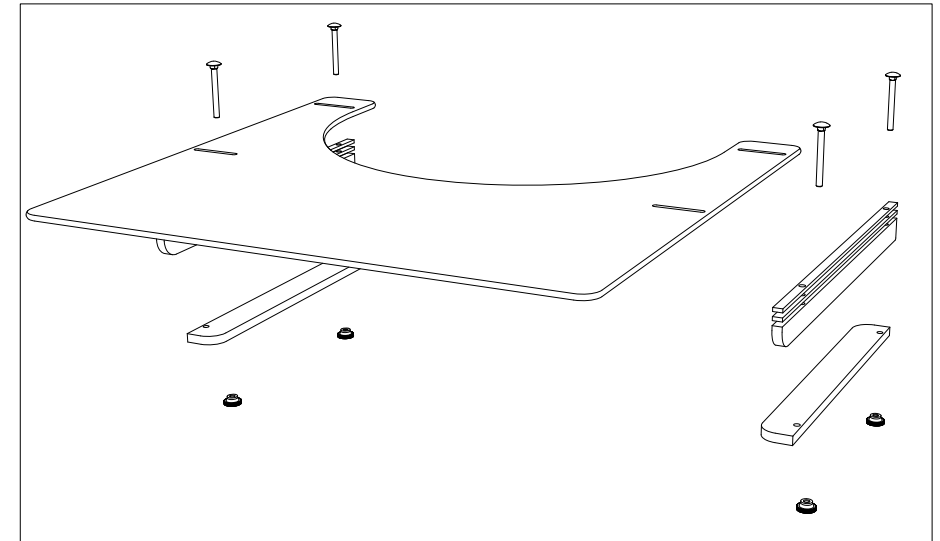

01)

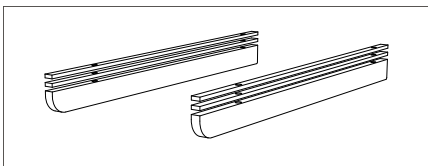


4 holding rails (L/B/H) á 400 x 10 x 4 mm  
2 holding rails (L/B/H) á 400 x 10 x 25 mm



Set 01) Depending on the wheelchair model a different amount of holding rails are used :


AS[01] / AS[01]RF = per armrest 1 x 25 mm high

02)

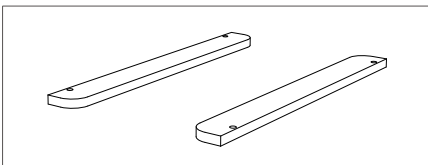

2 holding rails (L/B/H) á 350 x 38 x 10 mm

03)

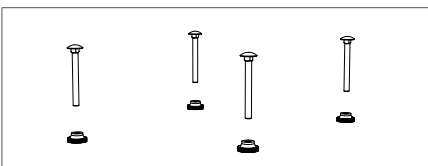

4 screws & 4 lock nuts

\*Serena II SW 38-45 only compatible with article 470061